

## PVC Shower Chair

Items: 89101, 89102, 89103, 89104, 89105, 89111  
89112, 89113, 89114, 89115, 89201, 89202  
89203, 89204, 89211, 89212, 89213, 89214

### Contents:

Partially assembled shower chair  
Four (4) caster wheels (2 braking & 2 non-braking) in brown box  
PVC back with mesh sling  
Six (6) ½" stainless steel screws

### Assembly Instructions:

#### Step 1

Remove partially assembled shower chair from shipping container/box.

#### Step 2

Insert the 4 casters into the opening of the fittings. Secure the casters by inserting and tightening ½" screws into the pre-drilled pilot holes on the fittings with a Phillips screwdriver.

#### Step 3

Insert the PVC back with mesh sling into the opening of the fittings. Secure the back with ½" screws.

NOTE: Casters and PVC mesh back must be checked and secured prior to using the shower chair.

### Care Instructions:

#### PVC Piping and Casters

Wipe down piping with non-abrasive cleanser. DO NOT use solvents. Restore luster with Johnson's Pledge or similar spray wax. If casters are used in wet environments, keep lubricated with white lithium paste or similar lubricant; remove hair and mop-thread build up.

#### Mesh Fabric (Sling)

If laundering is required, use only COLD water in large machine with regular detergent and one-cup bleach. Do not use softeners or solvents. Spin-dry only, machine heat will shrink mesh fabric. Hang damp fabric on frame and allow to air dry, returning to its original shape.

#### Footrest

Remove soap build-up from sliding surface with a soft bristle brush and soapy solution. Lubricate sliding mechanism with furniture spray wax to restore ease in movement.

#### Soft Seat

Wipe with non-abrasive disinfectant wash. Avoid contact with sharp or rough surfaces.

⚠ CAUTION: DO NOT use chair if casters are rusted and fail to move easily or if cracks occur in fittings.

