

SOLARIS
FREEDOM TO LIVE

*New Glove
& Gauntlet
Styles*

EXOTM
BY SOLARIS

*Off-the-Shelf Gradient
Compression Garments*

Table of Contents

2	Contact Information
3	Exo: A Full Range of Ready-to-Wear Solutions
4	Exo Quick Reference Chart
6	ExoStrong Upper Extremity <i>New Glove & Gauntlet Styles</i>
8	ExoStrong Lower Extremity
10	ExoSoft Upper Extremity <i>New Glove & Gauntlet Styles</i>
12	ExoSoft Lower Extremity
14	ExoSheer Lower Extremity
16	ExoFusion Lower Extremity
18	How to Read Exo Item Codes
20	Measuring Landmarks
21	Donning Instructions
22	Care Instructions, Indications and Contraindications
23	How to Order and Materials



Exo garments provide a full range of ready-to-wear solutions for your patients' unique needs. From ExoStrong ready-to-wear flat-knit garments for maximum containment to ExoFusion for focused compression over the foot, there's an Exo for everyone.

Why Choose Exo?



Like all of our ready-to-wear garments, all Exo orders placed before 5pm CST ship the same day.



Exo garments are easy to care for - machine wash and dry with a normal load of laundry.



ExoStrong is the complete line of ready-to-wear flat-knit garments and only costs a fraction of custom daywear.



Choose the style and structure that best accommodates your patients' unique needs.



Based on feedback from our customers, we've updated the target compression on ExoStrong garments. It still provides your customers with 20 - 30 mmHg, but now it will be easier for them to don.



















ExoSoft arm sleeves were the highest-rated circular-knit products by therapists and DMEs in single blind trials*.



ExoStrong™

ExoSoft™

Description	Ready-to-wear flat-knit garments that provide maximum containment while remaining comfortable - <i>all for a fraction of the cost of custom daywear.</i>	Circular-knit garments that provide comfortable gradient compression with excellent breathability.
Containment Level	 High	 Moderate
Available Compression Levels	20 – 30 mmHg	15 – 20 mmHg (LE Only) 20 – 30 mmHg
Color Options	 Black  Beige	 Black  Beige
Knit Structure	Flat-knit	Circular-knit
Available Styles	<div>  <p>Arm Sleeve UE-CG Sold Individually <i>Knit or Silicone Top</i></p> </div> <div>  <p>Glove UE-AC Sold Individually <i>DIPs to Wrist, PIPs to Wrist, or MCPs to Wrist</i></p> </div> <div>  <p>Gauntlet UE-BC Sold Individually</p> </div> <div>  <p>Thigh High LE-AG Sold Individually <i>Open Toe only Silicone Top only</i></p> </div> <div>  <p>Below Knee LE-AD Sold Individually <i>Open Toe only Knit or Silicone Top</i></p> </div>	<div>  <p>Arm Sleeve UE-CG Sold Individually <i>Knit or Silicone Top</i></p> </div> <div>  <p>Glove UE-AC Sold Individually <i>DIPs to Wrist, PIPs to Wrist, or MCPs to Wrist</i></p> </div> <div>  <p>Gauntlet UE-BC Sold Individually</p> </div> <div>  <p>Thigh High LE-AG Sold in Pairs <i>Closed Toe only Silicone Top only</i></p> </div> <div>  <p>Below Knee LE-AD Sold in Pairs <i>Open or Closed Toe Knit Top only</i></p> </div>

ExoSheer™

Discreet, sheer fabric offers efficient gradient compression and maximum breathability.

 Low

8 – 15 mmHg
15 – 20 mmHg
20 – 30 mmHg

 Black  Beige

Circular-knit




Thigh High
LE-AG
Sold in Pairs
Closed Toe only
Silicone Top only



Below Knee
LE-AD
Sold in Pairs
Open or Closed Toe
(8 - 15 mmHg Closed only)
Knit Top only

ExoFusion™

A comfortable sock that provides focused compression over the foot, making it easier to don than traditional stockings.

 Moderate (*Over Foot*)

20 – 30 mmHg (*Over Foot*)

 Black

Circular-knit



Below Knee
LE-AD
Sold in Pairs
Closed Toe only

ExoStrong™ Upper Extremity Garments

ExoStrong Arm Sleeve 54-221

A ready-to-wear arm sleeve designed for maximum support, comfort, and durability from the wrist to the axilla.



Designed with a functional bend at the elbow for improved comfort and range of motion.





Flat-knit structure provides maximum containment while remaining flexible.



Durable material helps resist snags and runs.



Made with fibers that combine a soft, tactile feel with moisture wicking and temperature control.

Options		
Colors		
Beige		Black
Proximal Options		
	Knit Top	
Compression		
20 - 30 mmHg		

ExoStrong Glove / Gauntlet 54-203 / 54-211

A comfortable, off-the-shelf glove / gauntlet that provides maximum containment.



Flat, reinforced edges help reduce irritation and rolling.



Tapered wrist design works in tandem with UE-CG garments to provide full arm coverage.



Gusseted web spaces provide flexibility while maintaining containment.



Made with fibers that combine a soft, tactile feel with moisture wicking and temperature control.

Options	
Colors	
Beige	
Black	
Compression	
20 - 30 mmHg	
Available Styles	
Full Finger Glove DIPs to Wrist	
Half Finger Glove PIPs to Wrist	
Quarter Finger Glove MCPs to Wrist	
Gauntlet MCPs to Wrist	

Pictured:
Quarter Finger
MCPs to Wrist

ExoStrong Arm Sleeve 54-221

All sizes in centimeters

Short		Arm Length: up to 43		
Description	Wrist Girth	Elbow Girth	Upper Arm Girth	Item Code
Small	15 - 17	24 - 27	27 - 30	54-221-44-XX-30X0
Medium	17 - 19	27 - 31	31 - 35	54-221-45-XX-30X0
Large	20 - 22	31 - 35	35 - 40	54-221-46-XX-30X0
X-Large	23 - 25	36 - 41	40 - 45	54-221-47-XX-30X0
Average		Arm Length: up to 46		
Description	Wrist Girth	Elbow Girth	Upper Arm Girth	Item Code
Small	15 - 17	24 - 27	27 - 30	54-221-54-XX-30X0
Medium	17 - 19	27 - 31	31 - 35	54-221-55-XX-30X0
Large	20 - 22	31 - 35	35 - 40	54-221-56-XX-30X0
X-Large	23 - 25	36 - 41	40 - 45	54-221-57-XX-30X0
Tall		Arm Length: up to 49		
Description	Wrist Girth	Elbow Girth	Upper Arm Girth	Item Code
Small	15 - 17	24 - 27	27 - 30	54-221-64-XX-30X0
Medium	17 - 19	27 - 31	31 - 35	54-221-65-XX-30X0
Large	20 - 22	31 - 35	35 - 40	54-221-66-XX-30X0
X-Large	23 - 25	36 - 41	40 - 45	54-221-67-XX-30X0

ExoStrong Glove / Gauntlet 54-203 / 54-211

All sizes in centimeters

Glove / Gauntlet	Length: N/A		
Description	Palm Girth	Wrist Girth	Item Code
Small	17 - 19	15 - 17	54-2XX-54-XX-3X00
Medium	19 - 21	17 - 19	54-2XX-55-XX-3X00
Large	22 - 24	20 - 22	54-2XX-56-XX-3X00
X-Large	25 - 27	23 - 25	54-2XX-57-XX-3X00

ExoStrong™ Lower Extremity Garments

ExoStrong Thigh High 54-107

Ready-to-wear gradient compression flat-knit garments provide maximum containment from the MTP to the groin.



Heel seam provides an anatomical fit and helps ensure garment is donned properly.



Flat, reinforced edges help reduce irritation and rolling.



Designed for a comfortable fit while standing to increase ROM and prevent bulking at the ankle.



Made with fibers that combine a soft, tactile feel with moisture wicking and temperature control.

Options	
Colors	
Beige	Black
Distal Option	
	Open Toe
Proximal Option	
	Silicone Top
Compression	
20 - 30 mmHg	

ExoStrong Below Knee 54-104

From the MTP to just below the knee bend, these ready-to-wear flat-knit garments offer maximum containment.



Heel seam provides an anatomical fit and helps ensure garment is donned properly.



Flat, reinforced edges help reduce irritation and rolling.



Provides a comfortable fit while standing and prevents bulking at the ankle.



Made with fibers that combine a soft, tactile feel with moisture wicking and temperature control.

Options	
Colors	
Beige	Black
Distal Option	
	Open Toe
Proximal Options	
	Knit Top
	Silicone Top
Compression	
20 - 30 mmHg	

ExoStrong Thigh High 54-107

All sizes in centimeters

Short	Foot Length: up to 17		Thigh Length: up to 71	
Description	Ankle Girth	Calf Girth	Thigh Girth	Item Code
Small	18 - 20	27 - 36	41 - 55	54-107-44-XX-3220
Medium	20 - 24	29 - 39	45 - 61	54-107-45-XX-3220
Large	24 - 28	32 - 43	50 - 66	54-107-46-XX-3220
X-Large	28 - 31	33 - 45	56 - 71	54-107-47-XX-3220
XX-Large	32 - 34	37 - 50	66 - 81	54-107-48-XX-3220
Average	Foot Length: up to 19		Thigh Length: up to 74	
Description	Ankle Girth	Calf Girth	Thigh Girth	Item Code
Small	18 - 20	27 - 36	41 - 55	54-107-54-XX-3220
Medium	20 - 24	29 - 39	45 - 61	54-107-55-XX-3220
Large	24 - 28	32 - 43	50 - 66	54-107-56-XX-3220
X-Large	28 - 31	33 - 45	56 - 71	54-107-57-XX-3220
XX-Large	32 - 34	37 - 50	66 - 81	54-107-58-XX-3220
Tall	Foot Length: up to 22		Thigh Length: up to 77	
Description	Ankle Girth	Calf Girth	Thigh Girth	Item Code
Small	18 - 20	27 - 36	41 - 55	54-107-64-XX-3220
Medium	20 - 24	29 - 39	45 - 61	54-107-65-XX-3220
Large	24 - 28	32 - 43	50 - 66	54-107-66-XX-3220
X-Large	28 - 31	33 - 45	56 - 71	54-107-67-XX-3220
XX-Large	32 - 34	37 - 50	66 - 81	54-107-68-XX-3220

ExoStrong Below Knee 54-104

All sizes in centimeters



Short	Foot Length: up to 17		Calf Length: up to 38	
Description	Ankle Girth		Calf Girth	Item Code
Small	18 - 20		27 - 36	54-104-44-XX-32X0
Medium	20 - 24		29 - 39	54-104-45-XX-32X0
Large	24 - 28		32 - 43	54-104-46-XX-32X0
X-Large	28 - 31		33 - 45	54-104-47-XX-32X0
XX-Large	32 - 34		37 - 50	54-104-48-XX-32X0
Average	Foot Length: up to 19		Calf Length: up to 41	
Description	Ankle Girth		Calf Girth	Item Code
Small	18 - 20		27 - 36	54-104-54-XX-32X0
Medium	20 - 24		29 - 39	54-104-55-XX-32X0
Large	24 - 28		32 - 43	54-104-56-XX-32X0
X-Large	28 - 31		33 - 45	54-104-57-XX-32X0
XX-Large	32 - 34		37 - 50	54-104-58-XX-32X0
Tall	Foot Length: up to 22		Calf Length: up to 44	
Description	Ankle Girth		Calf Girth	Item Code
Small	18 - 20		27 - 36	54-104-64-XX-32X0
Medium	20 - 24		29 - 39	54-104-65-XX-32X0
Large	24 - 28		32 - 43	54-104-66-XX-32X0
X-Large	28 - 31		33 - 45	54-104-67-XX-32X0
XX-Large	32 - 34		37 - 50	54-104-68-XX-32X0

ExoSoft™ Upper Extremity Garments

ExoSoft Arm Sleeve 53-221

This **#1 rated** circular-knit arm sleeve provides comfortable gradient compression and moderate containment.

-  Highest rated arm sleeve by therapists and DMEs in single blind trials.
-  Wide bands at the wrist and axilla help the sleeve stay in place and resist rolling.
-  Microfiber materials are comfortable and breathable while still providing containment for mild to moderate edema.
-  Soft and smooth materials are easier for patients to don.

Options	
Colors	
Beige	Black
Proximal Options	
 Knit Top	 Silicone Top
Compression	
20 - 30 mmHg	

ExoSoft Glove / Gauntlet 53-203 / 53-211

ExoSoft gloves / gauntlet are the ideal balance of comfort, containment, and breathability. It's so soft you have to try it to believe it!

-  Tapered wrist works with UE-CG garments to provide full arm coverage.
-  Lightweight microfiber fabric helps keep patients cool, dry, and comfortable.
-  Flat, reinforced edges help reduce irritation and rolling.
-  Gusseted web spaces provide flexibility while maintaining containment.

Options	
Colors	
Beige	Black
Compression	
20 - 30 mmHg	
Available Styles	
Full Finger Glove DIPs to Wrist	
Half Finger Glove PIPs to Wrist	
Quarter Finger Glove MCPs to Wrist	
Gauntlet MCPs to Wrist	

Pictured:
Quarter Finger
MCPs to Wrist

ExoSoft Arm Sleeve 53-221

All sizes in centimeters

Short	Arm Length: less than 43			
Description	Wrist Girth	Elbow Girth	Upper Arm Girth	Item Code
Small	15 - 17	24 - 27	27 - 30	53-221-44-XX-30X0
Medium	17 - 19	27 - 31	31 - 35	53-221-45-XX-30X0
Large	20 - 22	31 - 35	35 - 40	53-221-46-XX-30X0
X-Large	23 - 25	36 - 41	40 - 45	53-221-47-XX-30X0
Tall	Arm Length: more than 43			
Description	Wrist Girth	Elbow Girth	Upper Arm Girth	Item Code
Small	15 - 17	24 - 27	27 - 30	53-221-64-XX-30X0
Medium	17 - 19	27 - 31	31 - 35	53-221-65-XX-30X0
Large	20 - 22	31 - 35	35 - 40	53-221-66-XX-30X0
X-Large	23 - 25	36 - 41	40 - 45	53-221-67-XX-30X0

ExoSoft Glove / Gauntlet 53-203 / 53-211

All sizes in centimeters

Glove / Gauntlet	Length: N/A		
Description	Palm Girth	Wrist Girth	Item Code
Small	17 - 19	15 - 17	53-2XX-54-XX-3X00
Medium	19 - 21	17 - 19	53-2XX-55-XX-3X00
Large	22 - 24	20 - 22	53-2XX-56-XX-3X00
X-Large	25 - 27	23 - 25	53-2XX-57-XX-3X00

ExoSoft™ Lower Extremity Garments

ExoSoft Thigh High 53-107

When full leg coverage is needed, look no further than our comfortable ExoSoft Thigh High.



Wide silicone bands help reduce rolling and keep the stocking in place.




Soft and smooth materials are easier for patients to don.



Lightweight microfiber fabric helps keep patients cool, dry, and comfortable.



Heel pocket provides an anatomical fit and helps ensure the stocking is donned properly.

Options	
Colors	
Beige	Black
Distal Option	
	Closed Toe
Proximal Option	
	Silicone Top
Compression	
15 - 20 mmHg	
20 - 30 mmHg	

ExoSoft Below Knee 53-104

This circular-knit garment provides coverage to just below the knee crease, and looks as good as it feels.



Wide bands help reduce rolling and keep the stocking in place.






Soft and smooth materials are easier for patients to don.



Lightweight microfiber fabric helps keep patients cool, dry, and comfortable.



Heel pocket provides an anatomical fit and helps ensure the stocking is donned properly.

Options	
Colors	
Beige	Black
Distal Options	
	Open Toe
	Closed Toe
Proximal Option	
	Knit Top
Compression	
15 - 20 mmHg	
20 - 30 mmHg	

ExoSoft Thigh High 53-107

All sizes in centimeters

Thigh High	Length	Girths			
Description	Thigh	Ankle	Calf	Thigh	Item Code
Small	up to 73	18 - 20	27 - 37	41 - 56	53-107-54-XX-X120
Medium	up to 76	20 - 24	29 - 39	45 - 61	53-107-55-XX-X120
Large	up to 78	24 - 28	32 - 43	50 - 66	53-107-56-XX-X120
X-Large	up to 81	28 - 31	35 - 45	56 - 71	53-107-57-XX-X120

ExoSoft Below Knee 53-104



All sizes in centimeters

Below Knee	Length	Girths		
Description	Calf	Ankle	Calf	Item Code
Small	up to 40	18 - 20	27 - 37	53-104-54-XX-XX10
Medium	up to 43	20 - 24	29 - 39	53-104-55-XX-XX10
Large	up to 45	24 - 28	32 - 43	53-104-56-XX-XX10
X-Large	up to 48	28 - 31	35 - 45	53-104-57-XX-XX10

ExoSheer Thigh High 52-107

Sheer thigh high garments provide full leg coverage and feature a silicone lace band that helps prevent slipping throughout the day.




-  Wide silicone band helps reduce rolling and keeps the stocking in place.
-  Silky and elastic material is easier for patients to don.
-  Lightweight fabric provides maximum breathability.
-  Sheer fabric offers a natural look.

Options	
Colors	
Beige	Black
Distal Option	
	Closed Toe
Proximal Option	
	Silicone Top
Compression	
8 - 15 mmHg	
15 - 20 mmHg	
20 - 30 mmHg	

ExoSheer Below Knee 52-104

Sheer circular-knit stockings blend form and function with soft, silky fabric that's easy to put on and wear underneath any type of clothing.

-  Wide band helps reduce rolling and keeps the stocking in place.
-  Silky and elastic material is easier for patients to don.
-  Lightweight fabric provides maximum breathability.
-  Sheer fabric offers a natural look.

Options	
Colors	
Beige	Black
Distal Options	
	Open Toe
	Closed Toe
Proximal Option	
	Knit Top
Compression	
8 - 15 mmHg (Closed toe only)	
15 - 20 mmHg	
20 - 30 mmHg	

ExoSheer Thigh High 52-107

All sizes in centimeters

Thigh High	Length	Girths			
Description	Thigh	Ankle	Calf	Thigh	Item Code
Small	up to 73	18 - 20	27 - 37	41 - 56	52-107-54-XX-X120
Medium	up to 76	20 - 24	29 - 39	45 - 61	52-107-55-XX-X120
Large	up to 78	24 - 28	32 - 43	50 - 66	52-107-56-XX-X120
X-Large	up to 81	28 - 31	35 - 45	56 - 71	52-107-57-XX-X120

ExoSheer Below Knee 52-104

All sizes in centimeters

Below Knee	Length	Girths		
Description	Calf	Ankle	Calf	Item Code
Small	up to 40	18 - 20	27 - 37	52-104-54-XX-XX10
Medium	up to 43	20 - 24	29 - 39	52-104-55-XX-XX10
Large	up to 45	24 - 28	32 - 43	52-104-56-XX-XX10
X-Large	up to 48	28 - 31	35 - 45	52-104-57-XX-XX10

ExoFusion™ Foot Compression Garment

ExoFusion Foot 51-104

ExoFusion provides a focused 20 -30 mmHg compression over the foot. It can be worn alone or in conjunction with ReadyWrap, bandages, or layered under another compression garment.



Focused foot compression is easier for patients to don.



Made with fibers that combine a soft, tactile feel with moisture wicking and temperature control.



Looks and feels like a regular tube sock and fits inside shoes.



Can be paired with ReadyWrap to provide additional compression over the calf.

Options

Color

Black

Compression

20 - 30 mmHg (Over Foot)

20 - 30 mmHg


ExoFusion Foot 51-104

All sizes in centimeters

Short		
Foot Length: up to 21		
Description	MTP Girth	Item Code
Medium	20 - 24	51-104-45-02-3100
Large	24 - 28	51-104-46-02-3100
Tall		
Foot Length: up to 28		
Description	MTP Girth	Item Code
Medium	20 - 24	51-104-65-02-3100
Large	24 - 28	51-104-66-02-3100

Exo Item Codes

The Exo product line features a new, structured coding system. We know these item codes may take awhile to get used to, so if you have any questions give us a call at (855) 892-4140.

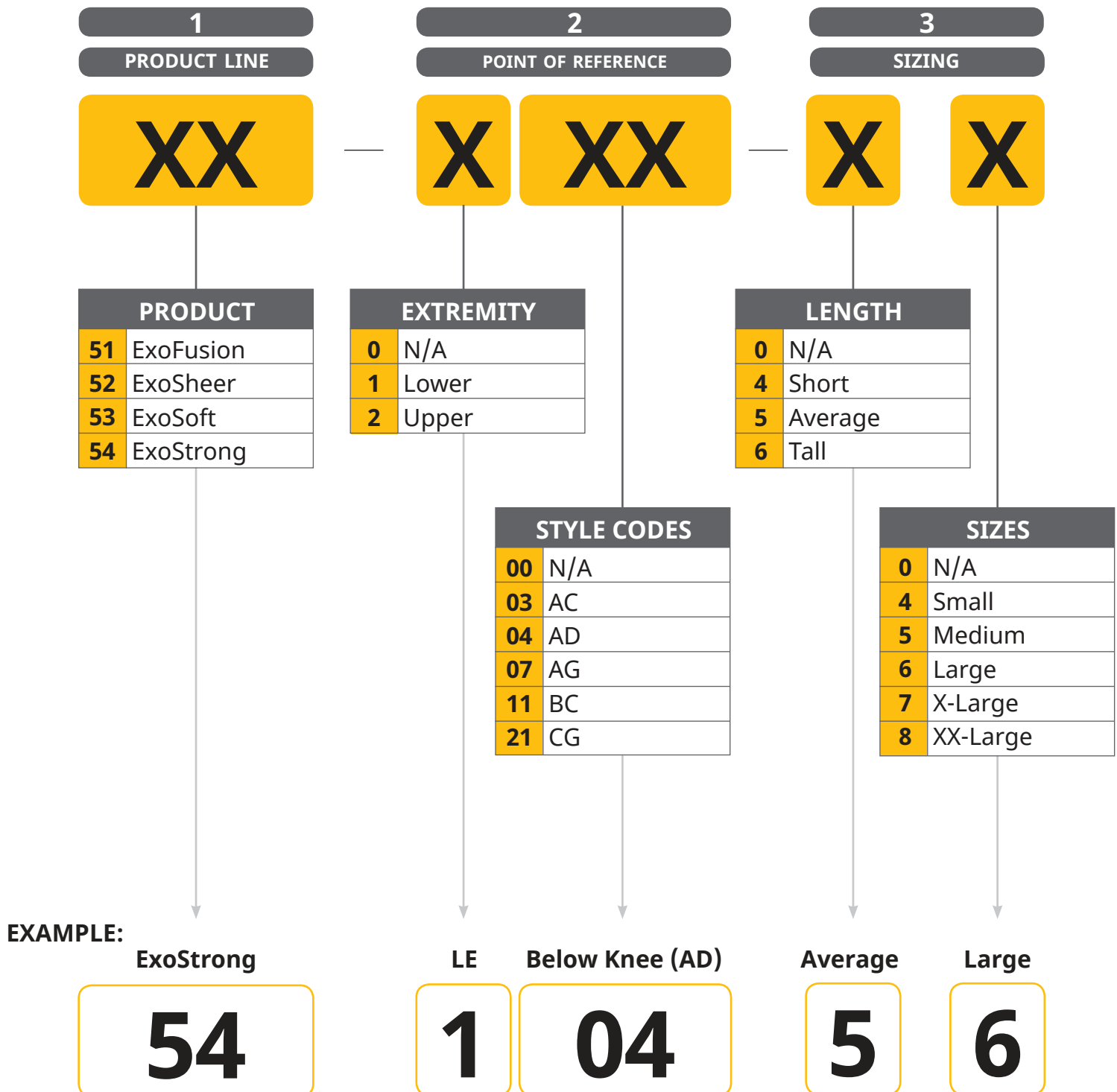

4
COLOR

XX

COLOR	
00	N/A
01	Beige
02	Black

Black

02

5
FEATURES

X X X X

COMPRESSION	
0	N/A
1	8 – 15 mmHg
2	15 – 20 mmHg
3	20 – 30 mmHg

20 – 30
mmHg

3

PROXIMAL OPTIONS	
0	N/A
1	Knit top
2	Silicone top

Knit
top

1

DISTAL OPTIONS	
0	N/A
1	Closed toe
2	Open toe
7	MCPs
8	PIPs
9	DIPs

Open
toe

2


REGION	
0	N/A
1	Left
2	Right

Bilateral

0

Measuring Landmarks

Measuring for Exo is easy. Simply use the landmarks below to determine the proper size garment.


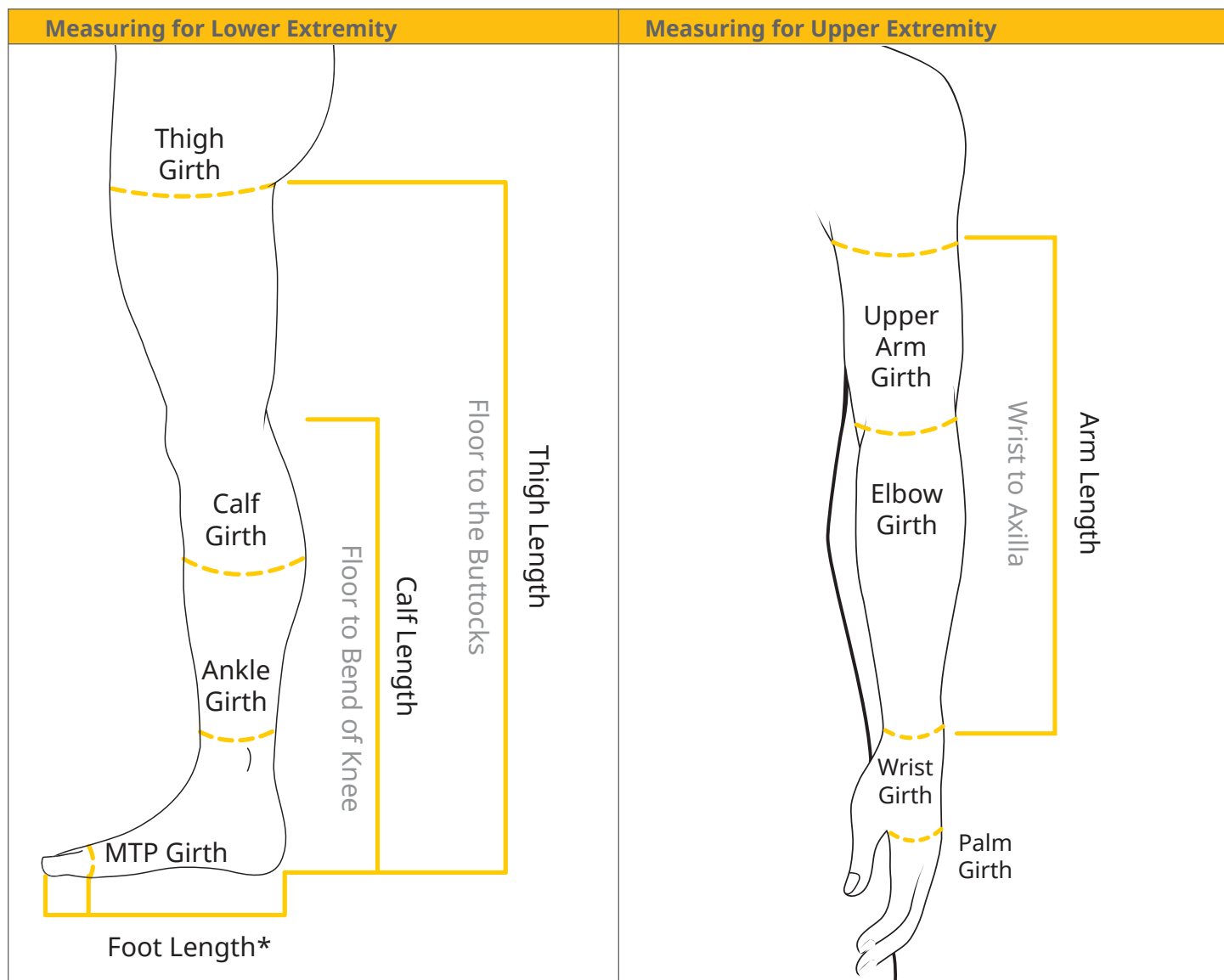


* ExoFusion measured from heel to toe.
ExoStrong measured from heel to MTP.

Donning Instructions

It is important to remove any jewelry before applying or removing your Exo garment. Avoid sticking fingernails or other sharp objects into the material as this may cause damage to the garment.

Lower Extremity (Knee Highs and Thigh Highs)	
Step 1	Step 2
<p>Fold the top half of your Exo garment inside out so that it is overlapping the lower half. Slide your foot inside until your heel is seated in the heel pocket or aligns with the heel seam.</p>	<p>Gently ease the top half of your garment up your leg. Smooth out any creases.</p>
	

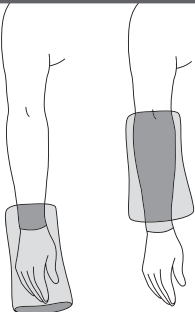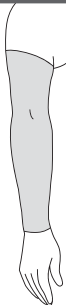
Arm Sleeve	
Step 1	Step 2
<p>Fold the top half of your Exo garment inside out so that it is overlapping the lower half. Slide your hand inside and gently pull the garment up to your elbow.</p>	<p>Ease the garment up your arm. Smooth out any creases.</p>
	

Glove and Gauntlet	
Step 1	Step 2
<p>If using an Exo arm sleeve, always put it on before using your Exo glove or gauntlet.</p>	<p>Slide your glove or gauntlet over your hand. Adjust for comfort around the fingers.</p>
	

Care Instructions

Washing

- Machine wash with mild detergent in warm water.
- DO NOT use bleach, fabric softeners, dryer sheets, or other additives.

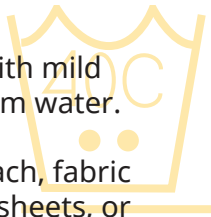

Drying

- Machine dry on permanent press.
- DO NOT iron.



Indications

- Edema

Contraindications

- Arterial insufficiency or degeneration
- Untreated congestive heart failure
- Systemic infection
- Impaired sensation
- Cognitive impairment
- Allergies or sensitivities to materials