NON-ADHESIVE FOAM DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Foam Dressings are highly absorbent and provide an ideal moist environment for healing. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time of up to seven days. These dressings are indicated for the treatment of moderate to heavy exuding wounds, and assist in protecting the wound, preventing bacterial contamination and maceration.

There are two layers, an absorbent foam that absorbs and retains exudates, and a waterproof yet vapor permeable film to provide protection (except 20463). The breathable film also allows excess gases to escape to allow moisture absorption. On style 20463 there is no waterproof film backing.

Vitále Foam Dressings are sterilized by radiation must not be resterilized.

INDICATIONS

Vitále® Silicone Foam Dressings are for single use only and can be used for the treatment of the following wounds:

Deep to
Superficial

- Pressure Sores
- Arterial Ulcers
- Venous Ulcers

Moderate to
Heavy Exuding
Wounds

- Diabetic Ulcers
- Skin Donor Site
- Second Degree Burns

CAUTION

Do not use with sodium hypochlorite or hydrogen peroxide solutions.

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitále® Foam Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean and rinse the wound in the usual manner.
2. Remove the release paper exposing dressing using sterile technique.
3. Place the Vitále Foam Dressing on the moist wound. The foam contact layer should cover the wound bed. Do not stretch the dressing while applying.
4. Cover Non-Adhesive Dressings appropriately.
5. Change dressing as directed, when the foam is saturated, or at least every 7 days.

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