COVER DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitale® Cover Dressings are absorbent, have an adhesive border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated for covering primary treatment dressings, and assist in protecting the wound, preventing bacterial contamination and maceration.

Cover Dressings have an absorbent gauze pad in the center, and a soft & strong backing made from spun-lace non-woven material. The adhesive border firmly holds the cover dressings in place, yet is easily removed at dressing change time.

Vitale® Cover Dressings are available in a selection of convenient sizes. They are sterilized by radiation and must not be resterilized.

INDICATIONS

Vitale® Cover Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:

Superficial or Deep

Moderate to Heavy Exuding Wounds

- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers
- Post-Operative Wounds
- Skin Donor Site
- Second Degree Burns
- Post-Surgery Sites

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitale® Cover Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

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