## COMPOSITE DRESSINGS

### INSTRUCTIONS FOR USE

#### PRODUCT DESCRIPTION

Vitale® Composite Dressings are absorbent, have an adhesive border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated for covering primary treatment dressings, and assist in protecting the wound, preventing bacterial contamination and maceration.

Composite Dressings have an absorbent gauze pad in the center, and a soft & strong backing made from spun-lace non-woven material. The adhesive border firmly holds the cover dressings in place, yet is easily removed at dressing change time.

Composite Dressings have an added a waterproof yet vapor permeable film to provide protection. They can be used on areas of the body affected by incontinence, to keep the wound clean, or over treated wounds while showering or bathing. The breathable film allows excess gases to escape.

Vitale® Composite Dressings are available in a selection of convenient sizes. They are sterilized by radiation and must not be resterilized.

#### INDICATIONS

Vitale® Composite Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:

**Superficial or Deep**

**Moderate to Heavy Exuding Wounds**

- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers
- Post-Operative Wounds
- Skin Donor Site
- Second Degree Burns
- Post-Surgery Sites

#### SIDE EFFECTS & INTERACTIONS

None have been observed to date.

#### STORAGE

Vitale® Composite Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

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