VITÁLE®

SILICONE COVER DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitale® Silicone Cover Dressings are absorbent, have a non-traumatic silicone border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated as a secondary dressing to cover primary treatments & dressings, or as a primary dressing on minor wounds, skin tears, etc. They assist in protecting the wound and preventing bacterial contamination or maceration.

Vitale® Silicone Cover Dressings have an absorbent non-adherent pad in the center, held firmly in place by the silicone, uniquely formed in an oval shape open to the passage of fluids into the pad. The silicone adhesive allows easy & safe removal of the dressing at change time, without trauma to the patient. The soft and strong backing is made from flexible polyurethane that provides a waterproof barrier. This breathable film allows gases, including water vapor, to escape the dressing.

Vitale® Silicone Cover Dressings are available in a selection of convenient sizes. They are sterilized by ethylene oxide and must not be resterilized.

INDICATIONS

Vitale® Silicone Cover Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:

- | | | |
|---|-------------------------------|---|
|  | Superficial to Full Thickness | <ul style="list-style-type: none">• Pressure Sores• Arterial Ulcers• Venous Ulcers |
|  | Dry to Highly Exuding Wounds | <ul style="list-style-type: none">• Diabetic Ulcers• Post-Operative Wounds• Skin Donor Site• Second Degree Burns• Post-Surgery Sites• Minor Wounds & Lacerations |

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitale® Silicone Cover Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

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