AVANOS*

CORFLO*
2 PORT "Y"
EXTENSION SET

Instructions for Use

AVANOS* CORFLO* 2 Port “Y” Extension Set

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Indication for Use

AVANOS* CORFLO* 2 Port “Y” Extension Sets are intended for delivery of enteral nutrition, water and medications through a feeding tube.

⚠ Warning: Not for IV use. For enteral use only. Not for infusion of parenteral solutions.

Instructions

1. Cap access port.
2. Insert enteral administration set connector into uncapped side of “Y” Extension Set.
3. Open clip clamp and prime tubing with formula. Close clip clamp.
4. Insert “Y” Extension set orange connector into feeding tube. Open clip clamp.
5. To irrigate, aspirate or administer medications, stop formula administration.
6. The administration set tubing must be closed off before opening access port:
 - A. Close roller clamp, or
 - B. Pinch administration set tubing closed.
7. To irrigate feeding tube using a Luer slip or Luer lock syringe:
Insert Luer syringe tip directly into access port.
8. To irrigate feeding tube using a cath tip syringe:
Insert Luer tip of syringe adaptor into access port. Insert syringe tip into syringe adaptor, hold firmly.

Note:

- Vigorous pressure should not be used during irrigation.
 - The feeding tube should be flushed with water whenever formula is interrupted and BEFORE and AFTER medication administration.
 - The feeding tube should be irrigated routinely every four hours.
9. After irrigation, close access port and open administration set tubing to allow formula flow:
 - A. Open clip clamp, or
 - B. Release pinched tubing. Restart formula administration. Proceed with feeding.

Single Patient Use	Non Sterile	Product is not made with DEHP as a Plasticizer	Product is not made with BPA
Not made with natural rubber latex	RX Only	Caution	Consult instructions for use

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*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2019-07-25 15-M1-321-01 / 70215480
Distributed in the USA by Avanos Medical Sales, LLC,
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue,