



QUICK START GUIDE

Wired Accessibility Switches

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase.



Made with recycled content

EN June 18, 2021 1:59 PM



Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.



Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.



To get started:

1. Plug switch into any switch-adapted device with a standard 3.5-mm / 1/8-in TS (mono) jack.
2. Once the activation area is pressed (some switches are activated with a pull or squeeze), any attached device will activate.



ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.




3

Device Overview

Devices shown below are examples of wired accessibility switches. More information for your wired accessibility Switch can be found at www.ablenetinc.com.



Big Buddy Button

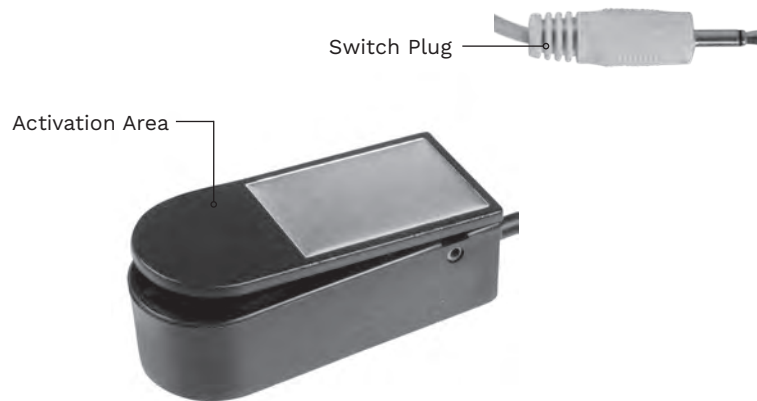

Micro Light Switch



Grasp Switch



Create printable picture symbol overlays for your device with the *FREE* AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

