



Magic³ Go[®]

HYDROPHILIC INTERMITTENT CATHETER
with CleanGlide[™] Technology

INSTRUCTIONS FOR USE

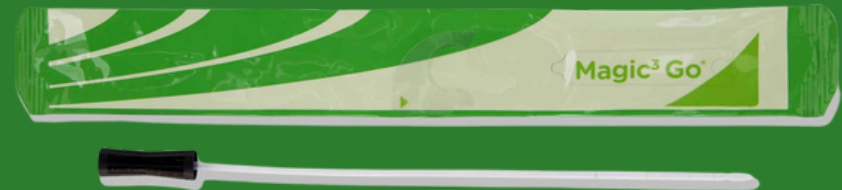

For use with 16" male length catheters
and 10" pediatric length catheters

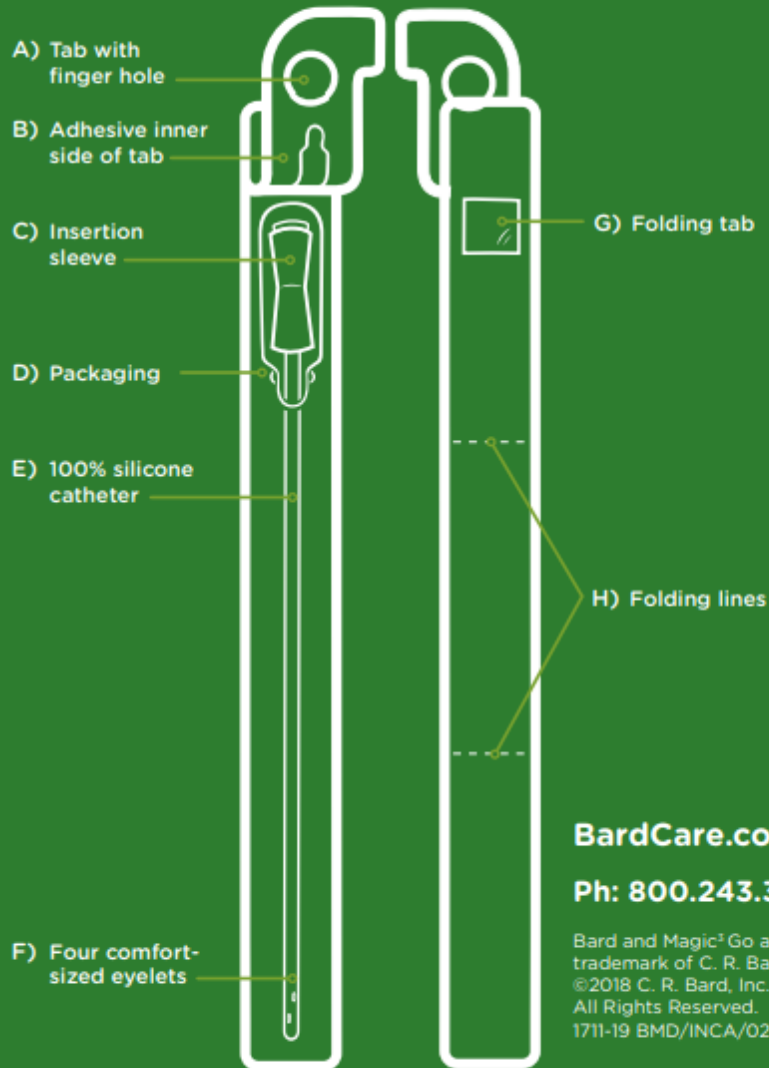

BardCare.com

Ph: 800.243.3315

Bard and Magic³ Go are registered
trademark of C. R. Bard, Inc.
©2018 C. R. Bard, Inc.
All Rights Reserved.
1711-19 BMD/INCA/0218/0207



Instructions for use

Review the self-catheterization procedure with a healthcare professional.

1. Always wash hands prior to use.
2. Open the catheter package by peeling the tab upwards with the aid of the finger hole.
3. If desired, use the adhesive label(s) to hang the package on a nearby dry vertical surface while preparing to catheterize.
4. Clean the opening of the urethra—and the surrounding area. Wash your hands again.
5. Hold the insertion sleeve with your dominant hand and squeeze it to grip the catheter shaft as you remove the catheter from the pack. Next, hold the catheter funnel above the insertion sleeve with your other hand and slide down the shaft, stopping at about six inches from the tip. Release the funnel. Using the insertion sleeve to hold the catheter firmly, gently pass the tip of the catheter firmly into your urethra until the insertion sleeve nears the meatus. Repeat until urine starts to flow.
6. Keep the catheter steady until urine stops flowing. When urine stops flowing, slowly withdraw the catheter, stopping if flow starts again, until the last few drops have drained. Check the color, odor and clarity of the urine. Any changes may need to be reported to a health care professional.

Disposal

7. Place the catheter back into the package and dispose in accordance with accepted medical practice and applicable local, state and federal laws and regulations. Do not flush down the toilet.
8. Wash hands.

3.


4.

5.

6.

7.

8.

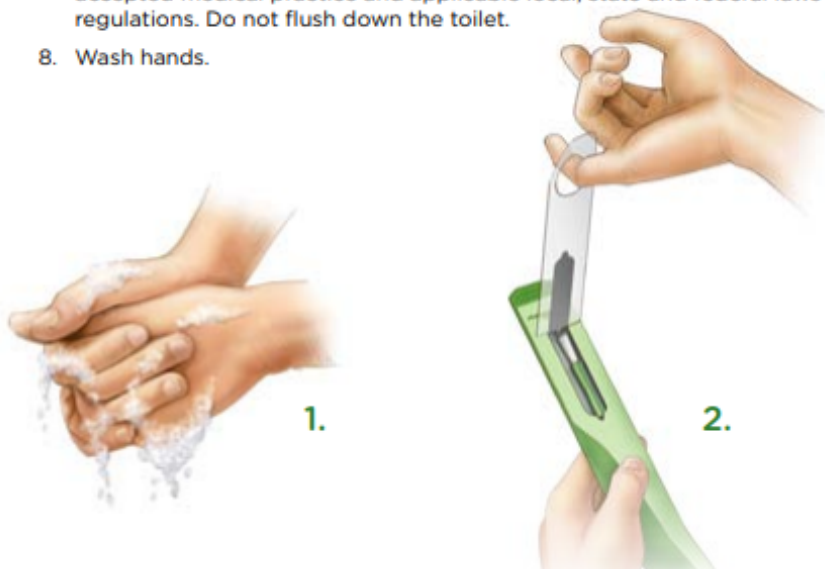

1.



2.

