GUARDIAN[®]

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**Aluminum Commode
Shower Chair,
Self Propelled**

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GENERAL WARNINGS

To ensure the safe assembly and operation of the product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury and/or damage may occur.
- DO NOT use if components are missing or damaged.
Contact Medline at 1-800-MEDLINE(633-5463) | 1-800-268-2848.
- DO NOT exceed the weight capacity of 275 lbs (124 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the frame of the product.
- DO NOT stand or apply excess pressure on the footrests. When getting in or out of the chair, ensure the footplates are in the UP position.
- DO NOT attempt to lift the product by the armrests or any detachable component.
- DO NOT place hands or fingers near any moving part.
- DO NOT transfer in or out of the chair on your own unless you have sufficient upper body strength, balance and agility.
- DO NOT lean forward out of the chair any farther than the length of the armrests.
- DO NOT attempt to reach for objects if doing so means you move forward in the seat.
- DO NOT shift your weight or sitting position toward the direction you are reaching as the chair may tip over.
- DO NOT attempt to lean over the top of the backrest cushion to reach objects.
- DO NOT tip the chair without assistance.
- DO NOT lean the chair back as this may cause the chair to tip over. If trying to reach objects, reach back only as far as your arm will extend without altering your seating position.
- DO NOT modify the chair. Examples; drilling holes, grinding, welding parts, using non-authorized/non-approved parts. Such modifications may change the chair's structure and center of balance which may increase the risk of serious injury to the user and void the warranty.
- DO NOT use this chair in an outdoor setting.

ASSEMBLY INSTRUCTIONS

READ ALL INSTRUCTIONS CAREFULLY BEFORE USING THIS CHAIR.

1. Mounting the backrest assembly: Unscrew both locking knobs from the backrest frame assembly. Insert the backrest frame assembly into the frame and secure with the locking knobs. Lower the armrests into position for use. **(Figure 1)**
2. Mounting the seat: Place the seat onto the seat frame. Firmly press down until each clip snaps into place. Note: Ensure the recessed portion is located at the rear. **(Figure 2)**
3. Mounting the footrests: Insert the footrests into the frame tubes and swivel the footrest forward. The footrest should click into place. **(Figure 3)**

- | | |
|-----------------|---------------------|
| 1. Push handle | 5. Wheel locks |
| 2. Back cushion | 6. Rear Wheels |
| 3. Armrest | 7. Footrests |
| 4. Seat | 8. Steering castors |

ADJUSTMENTS

Adjusting the rear wheel height

If adjusting the rear wheel height, you must also adjust the front wheel height to ensure these are level.

1. Push the quick-release button on the rear wheel and pull it out to remove it from the frame.
2. Using the allen key and tool provided, remove the rear wheel axle bracket from the frame.
3. Once removed, adjust the bracket onto the desired position.
4. Mount the bracket onto the frame using the allen key and tool provided. Ensure the screws are fully tightened. Repeat until both sides are adjusted as desired.
5. Install the rear wheels onto the frame by pushing the quick-release button and inserting it through the axle bracket.

NOTE: Ensure the quick-release axles are locked into place prior to use. The axles are locked when the quick-release button is fully out and the ball bearings are fully protruding through the inside of the frame. If the rear wheels do not lock into place, DO NOT use product and contact Medline.

Adjusting the castors

1. Remove the front castor bolt by utilizing the allen key and tool provided.
2. Adjust the castor to the desired height so that the rear wheels and castors are leveled.
3. Lock the castor using the bolt and nut. Repeat until both castors have been adjusted to the same height. **(Figure 4)**

Adjusting the footrests

1. Unscrew the bolt and nut from the rear side of the leg rest frame.
2. Set to desired height and screw on bolt and nut.
3. Repeat until both footrests have been adjusted to the desired height. Ensure the screws are fully tightened. **(Figure 5)**

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

SHOWER COMMODE USAGE

Locking Castors

1. Step down on the tab to lock the castor in place.
2. To release, lift up from underneath the tab. **(Figure 6)**

Wheel Locks

1. Push lever forward to apply wheel lock.
2. Pull lever toward rear of chair to release wheel lock. **(Figure 7)**

Commode Feature

1. Always lock the locking castors and rear wheels prior to using the chair.
2. Remove the lid from the bucket and push the bucket into place from the rear side of the chair.
3. Remove the padded seat lid cut-out.
4. After use, pull bucket out from the rear side. Empty the contents and clean immediately.
5. Replace the padded seat lid cut-out.

Over the toilet feature

1. Remove the bucket and lid from the rear side.
2. Remove the padded seat lid cut-out.
3. Raise the toilet seat and lid.
4. Roll the commode over the toilet and lock the castors and rear wheels.

Shower Chair Feature

1. Pull the bucket and lid out from the rear side.
2. Remove padded seat lid cut-out.
3. When using the chair in the shower, ensure the locking castors and wheel locks are engaged.

SAFETY INSTRUCTIONS

Reaching/leaning/bending-forward or sideways

- Position the chair as close as possible to the object.
- When reaching forward make sure the front castors are pointing in a forward position **(Figure 8)**. This extends the wheel base and will help stabilize the chair.
- When reaching sideways, position the casters to provide the greatest stability side to side. **(Figure 9)**
- Make sure the wheel locks are engaged before sitting down, standing up, reaching, or using as a commode.
- DO NOT attempt to reach for objects if doing so means you must move forward in the seat. DO NOT shift your weight toward the direction you are reaching.
- DO NOT attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.
- DO NOT shift your weight or seat position toward the direction you are reaching.

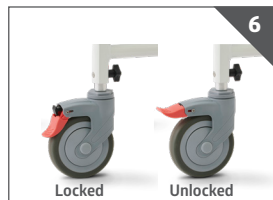
Figure 6

Figure 7

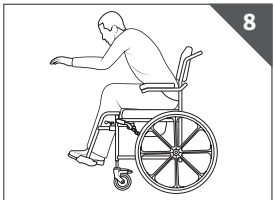
Figure 8

Figure 9

SAFETY INSTRUCTIONS (CONTINUED)

Reaching / leaning –backward

- Position the chair as close as possible to the object. Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the chair. **(Figure 10)**
- DO NOT lean back over the top of the back cushion. Reach back only as far as your arm will extend without changing your position in the seat. **(Figure 10)**

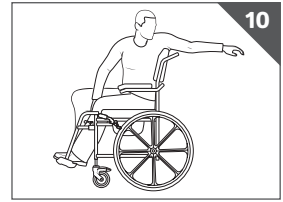
Figure 10

Transferring into and out of the chair

Transferring to and from the chair is a significant task that requires balance, agility and upper body strength. Practice with assistance prior to attempting transfers.

- Position the chair as close as possible.
- Position both castors parallel to the seat you are transferring to.
- Make sure all wheel castor locks are engaged.
- Ensure the armrests and footrests are moved out of the way. **(Figure 11, 12)**

Figure 11

Figure 12

Obstacles

Obstacles such as floor threshold strips, uneven floors, cords or items on the floor, or cracks in pavements can damage the chair and may cause loss of control or tipping over. Scan your path and avoid any areas that may cause instability.

SAFETY INSPECTION AND MAINTENANCE

Prior to use, ensure the following:

- There are no loose, broken or missing components.
- All screws, nuts and/or bolts are tightened.
- The wheels and wheel locks are not worn, broken or missing. Replace any missing or damaged parts before use.
- Always ensure the wheel locks are properly engaged during use.
- User evenly distributes weight between the arm supports when transferring in and out of the commode.

CLEANING

Clean after each use.

- Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
- DO NOT use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Medline guarantees for the original purchaser, that the frame of the chair will be free from defects in materials for the life of the chair.

Medline also guarantees the parts on the chair to be free from defect in materials and workmanship for a period of one (1) year from the date of purchase by the original purchaser.

This warranty does not cover device failure due to owner misuse, negligence or normal wear and tear. All warranty claims must include the product in questions unless otherwise authorized by Medline. For more information about the product or warranty, call Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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