

Anchor Bathing Chair Tub Stand User Manual

Thank you for purchasing the
Anchor Tub Stand
AN2200

Please read these instructions carefully before
assembling or using the Anchor Tub Stand.


Save these instructions for future reference.

Note:

All adjustments are tool-free for your convenience

ASSEMBLY

Assembly

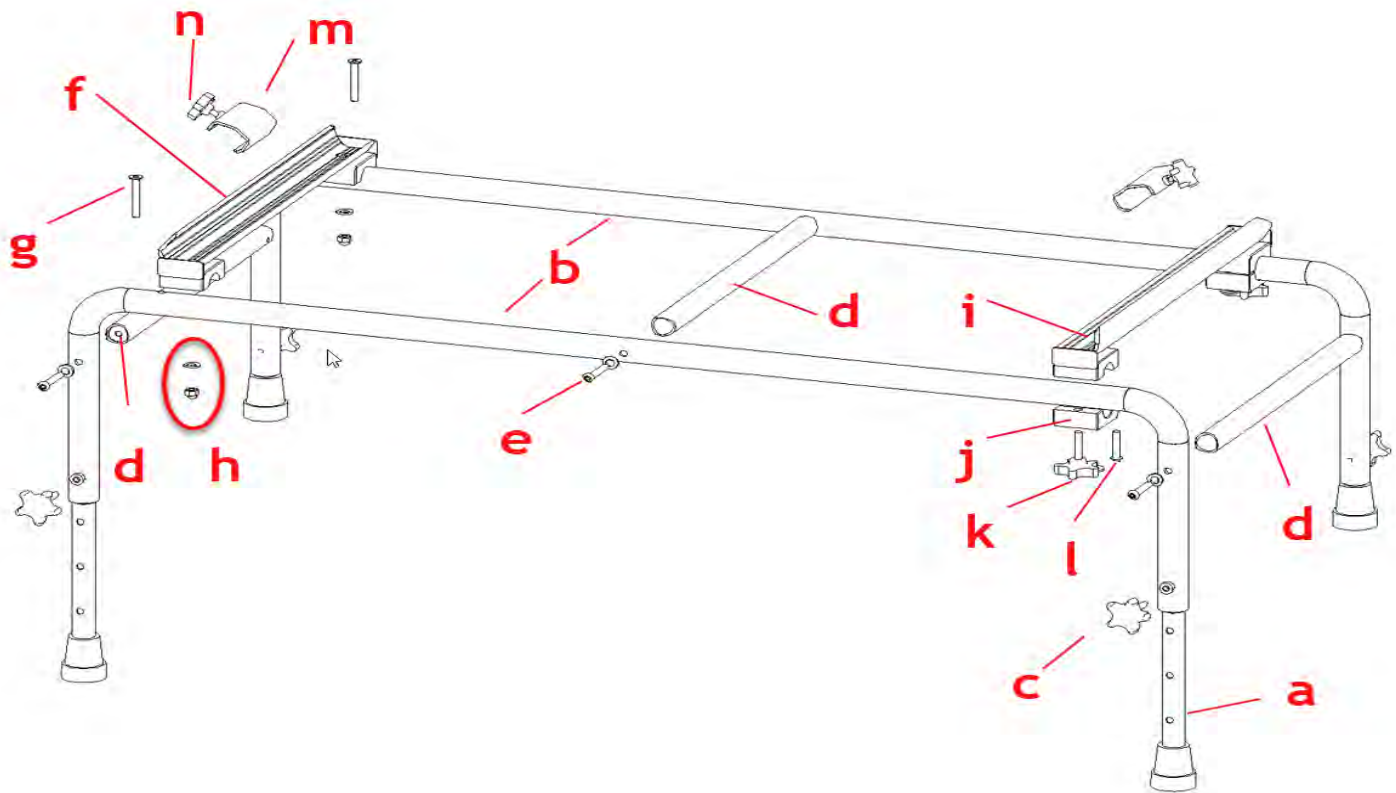

1. Insert all four **inner legs (a)** into the bottom of the **long rails (b)** using four **knobs (c)**.
2. Align the **rails (b)** so the top holes are on the same side.
3. Connect all three **support bars (d)** to the **long rails (b)** using the **longer Allen screws** and **curved washers (e)**.

Mounting Brackets

The fixed and sliding mounting brackets must both be positioned with the higher side facing outward from the side rails and angled towards each other.


4. Place **fixed mounting bracket (f)** on the top holes of the **long rails (b)**.
5. Secure the **fixed mounting bracket (f)** using **flat screws (g)**, along with **curved washers** and **lock nuts (h)** beneath the **rails (b)**.
6. Place the **sliding mounting bracket (i)** at the other end of the **long rails (b)**.
7. Secure the **sliding mounting bracket (i)** with double-holed **mounting clamps (j)** beneath the **rails (b)** using **knobs (k)** on the outward-facing hole and **shorter Allen screws** with **flat washers (l)** on the inward-facing hole.
8. Ensure all screws and knobs are tightened securely for stability

Placement and Adjustment

The Anchor Tub Stand provides additional height, making it easier to get in and out of the bathtub.

Simply position the sliding mounting bracket to correspond with the desired position and angle of the Anchor Bath Chair legs.

1. Place the rear leg of the Anchor bath chair into the **fixed mounting bracket (f)**.
2. Move the **sliding mounting bracket (i)** to proper position, ensuring it is aligned for the front of the chair.
3. Place the front leg of the Anchor bath chair into the **sliding mounting bracket (i)**.
4. Once the chair is in the desired position, tighten the **knobs (k)** to secure the bath chair in place.
5. Place the **securing clamps (m)** with **knobs (n)** facing outward over Anchor legs.
6. Tighten **knobs (n)** to secure the chair legs firmly in the tub stand.

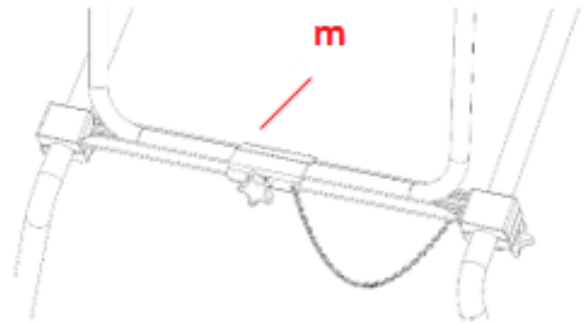

Make sure everything is securely tightened before use to ensure stability and safety.

Limited 2 Year Warranty

This warranty is solely for the **benefit** of the original consumer purchaser and is in effect from date of purchase.

Circle Specialty Inc. warrants that the Anchor Bath Chair Accessories will be free from defect for a period of two (2) year.

The warranty does not extend to non-durable components, such as, rubber accessories and grips.

This warranty does not cover device failure due to owner misuse or negligence. It only applies when product is used according **to the specified conditions and for the** intended purposes.

If within the warranty period any such product shall be proved defective, such product will be repaired or **replaced at Circle Specialty's** option.

If you have a question about your Circle Specialty wheelchair or this warranty, please contact an authorized dealer.

