NON-ADHERENT OIL EMULSION DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Non-Adherent Oil Emulsion Dressings are semi-occlusive primary dressings made of gauze mesh impregnated with an oil emulsion blend. They maintain a moist wound environment to support healing while allowing exudates to pass through. They cling & conform to body contours. Use beneath a secondary dressing, such as cover dressings, composite dressings or hydrocolloid dressings.

The oil emulsion blend helps prevent dressing adherence, while forming a semi-occlusive barrier, allowing excess exudate in the wound to pass through to be absorbed by the secondary dressing. The dressings may be cut or folded to fit the wound cavity or wrapped around extremities. Vitále® Non-Adherent Oil Emulsion Dressings are sterilized by exposure to gamma radiation and must not be resterilized.

INDICATIONS

Vitále® Non-Adherent Oil Emulsion Dressings can be used as a primary dressing for wounds which are:

- | | | |
|--|-------------------------------|---------------------------------------|
|  | Superficial to Full Thickness | • Incision sites |
|  | Dry to Highly Exuding Wounds | • Donor sites & skin graft sites |
| | | • Abrasions & minor burns |
| | | • Venous & pressure ulcers |
| | | • Staples, suture lines & lacerations |

CAUTION

- Do not use in cases of known sensitivity to the dressing & its components.
- Wounds should be reviewed and appropriately managed, under the direction of a medical practitioner.

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitále® Non-Adherent Oil Emulsion Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean and rinse the wound, using saline or appropriate wound cleanser.
2. Skin around the wound should be clean & dry.
3. Using aseptic techniques, remove dressing from package and apply to moist wound bed. Dressing may be cut to size.
4. Cover with a secondary absorbent dressing, such as cover or composite dressing or hydrocolloid dressing.
5. Dressing change as directed, when strike through occurs, or at least every 7 days.

PRODUCT LINE

CATALOG NUMBER	DESCRIPTION	SIZE	UNITS/BOX
58303	Vitále® Non-Adherent Oil Emulsion Dressing	3"x 3" (7.5 cm x 7.5 cm)	50
58308	Vitále® Non-Adherent Oil Emulsion Dressing	3" x 8" (7.5 cm x 20 cm)	24
58316	Vitále® Non-Adherent Oil Emulsion Dressing	3" x 16" (7.5 cm x 40 cm)	36

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