

The Arjo sling sizing and measurement system considers both the height (where applicable) and width of the patient to determine the correct sling size.

The system includes sizing tables (with size colors) that matches patient measurements with sling sizes. The tables and instructions are specific to the different sling types and can also be found in the Instructions for Use (IFU).

There are many different Arjo sling IFUs, and in this guide, all the sizing charts and measurement instructions have been gathered in the same document to support efficient sling sizing.

This guide addresses sling sizing and measurement only and does not replace the sling IFUs. Always refer to the IFU when using Arjo slings.

How do I know that I selected the right size?

Once the sling has been fitted to the patient, there are a number of key indicators to help verifying that the right size has been selected.

Detailed guidance can be found in the section “Applying the sling” in the sling IFUs Instructions;

1. Find the article number of the sling (e.g. on the product label).



2. Identify the correct sizing chart in the product list.
3. Follow the instructions in the chart for how to measure height and width of the patient and check the chart to find the sling size.

PRODUCT LIST		
Article number	Sling name	Chart name
MAA2000M	Clip sling – unpadded legs	“Regular passive clip slings”, page 4
MAA2040M	Clip in situ sling – unpadded legs	
MAA2070M	Clip bathing in situ sling – unpadded legs	
MAA4000M	Clip sling – padded legs	
MAA4050M	Clip comfort sling – padded legs	
MAA4060M	Clip bathing sling – unpadded legs	
MAA4061M	Clip bathing sling – padded legs	
MFA1000M	Disposable clip sling	
MAA4031M	Clip toilet sling – padded legs	
MAA4100M	Clip sling – padded legs and extended legs	“Extended legs clip slings”, page 5
MAA4160M	Clip bathing sling – unpadded and extended legs	
MFA2100	Disposable loop sling	“Passive loop slings”, page 6
MLA2000	Loop sling – padded legs	
MLAAS2000	Loop in situ sling – padded legs	
MLA3000	Loop sling – padded legs, without head support	
MLA4060	Loop bathing sling – unpadded legs	
MLA4031	Loop toilet sling – padded legs	
MLA4531	Loop toilet sling – padded legs, without head support	
MAA4020M	Paediatric clip sling	“Pediatric sling”, page 7
THA6i	Loop hammock sling – 6 straps	“Hammock slings”, page 8
THAi	Loop hammock sling – 4 straps	
TIR	Loop quick fit sling – without head support	“TIR sling”, page 9
MAA2050M	Amputee clip in situ sling	“Double amputee clip slings”, page 10
MAA2080M	Amputee clip bathing in situ sling	
MAA2090M	Amputee clip sling	
MAA2091M	Amputee clip toilet sling	
MAA4040M	Amputee clip sling - “hook & loop” fastener	
MAA7090M	Amputee clip bathing sling	

PRODUCT LIST		
Article number	Sling name	Chart name
MAA4070M	Amputee right leg clip sling	"Single leg amputee clip slings", page 11
MAA4080M	Amputee left leg clip sling	
MLA7000	Amputee loop hammock sling	"Hammock amputee sling", page 12
MAA8000	Bariatric loop comfort sling	"Bariatric slings", page 13
MAA8010	Bariatric loop sling	
MFA8010	Disposable bariatric loop sling	
MAA8020	Bariatric loop hammock sling - divided legs	
MAA8030	Bariatric loop hammock sling	"Standing transfer and walking slings", page 14
TEM	Loop walking sling	
MAA3071M	Sara clip standing sling – flat	
MFA3000	Sara Flex disposable standing sling	
TSS.500 – 504	Sara Flex standing sling	
TSS.500S-1C - 503S-1C	Sara Flex standing sling – separate belt with one buckle	
TSS.500W - TSS.503W	Sara Flex wipeable standing sling	
TSS.511	Sara Flex seated transfer sling	
KKA5120	Sara Plus standing sling	
KKA5130M	Sara Plus walking sling	
KKA5370	Sara Plus standing sling – extra postural support	
MFA4000	Sara Plus disposable standing sling	
KKA6000	Sara Plus wipeable standing sling	
MAA5000	Loop walking jacket	"Loop walking sling", page 15
MAA1150	Loop stretcher sling – commode aperture	"Soft stretcher slings", page 16
MAA1151	Loop stretcher sling	
MAA1152	Loop bathing stretcher sling – commode aperture	
MFA5000	Disposable loop stretcher sling	

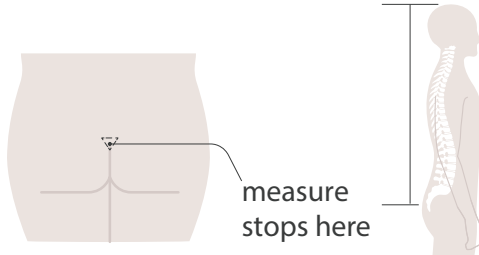
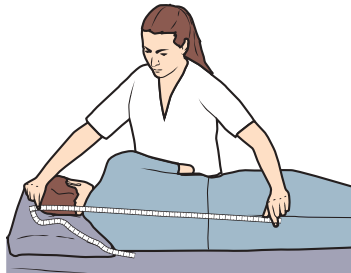
Regular passive clip slings

Sizing chart

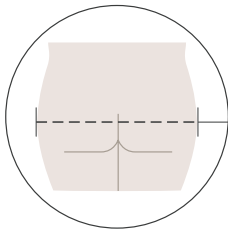
Sling Models

MAA2000M, MAA2040M,
MAA2070M, MAA4000M,
MAA4060M, MAA4031M,
MAA4061M, MFA1000M,
MAA4050M*

Top of the head to coccyx

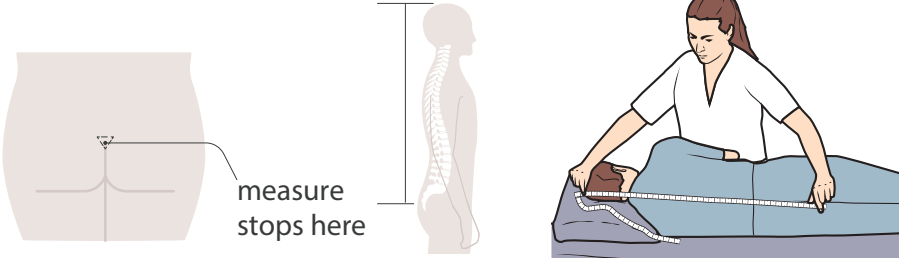
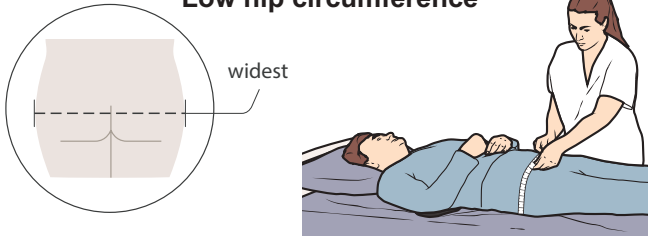
Low hip circumference




cm (in)	60-65 (23.5-25.5)	66-73 (26-29)	74-81 (29-32)	82-89 (32.5-35)	90-95 (35.5-37.5)	96-103 (38-40.5)
70-80 (27.5-31.5)	XS	S	M	L	LL	XL
81-93 (32-36.5)	S	S	M	L	LL	XL
94-106 (37-41.5)	M	M	M	LL	LL	XL
107-125 (42-49)	M	L	L	LL	XL	XL
126-136 (49.5-53.5)	L	LL	LL	LL	XL	XL
137-155 (54-61)	XL	XL	XL	XL	XL	XXL
156-220 (61.5-86.5)* * is limited to 200 cm / 78.5 in	/	/	XXL	XXL	XXL	XXL

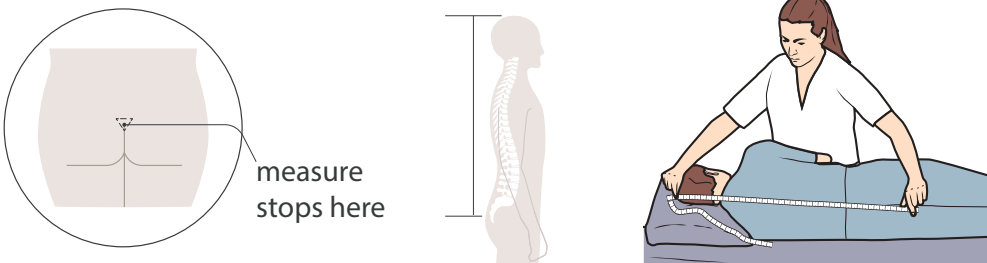
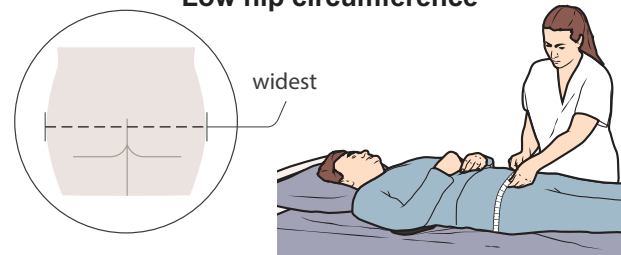
Extended legs clip slings

Sizing chart

Sling Models		Top of the head to coccyx						
MAA4100M, MAA4160M								
Low hip circumference		cm (in)	60-65 (23.5-25.5)	66-73 (26-29)	74-81 (29-32)	82-89 (32.5-35)	90-95 (35.5-37.5)	96-103 (38-40.5)
		87-100 (34-39.5)	S	S	M	L	LL	XL
		101-117 (40-46)	M	M	M	LL	LL	XL
		118-130 (46.5-51)	M	L	L	LL	XL	XL
		131-145 (51.5-57)	L	LL	LL	LL	XL	XL
		146-175 (57.5-69)	XL	XL	XL	XL	XL	XXL
		176-240 (69.5-97.5)	/	/	XXL	XXL	XXL	XXL

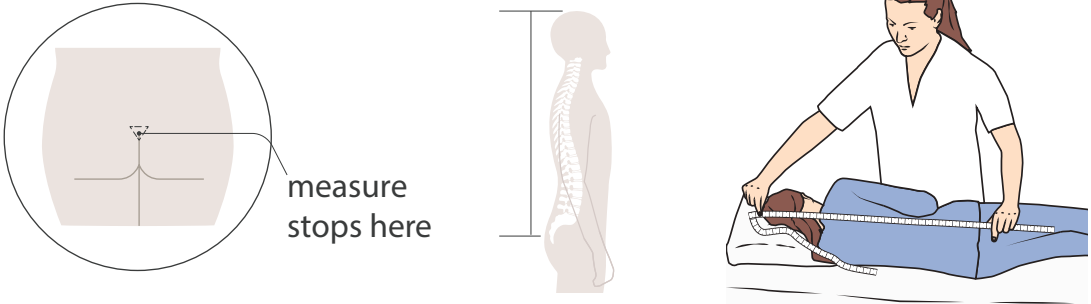
Passive loop slings

Sizing chart

Sling Models		Top of the head to coccyx					
MLA2000, MLAAS2000, MLA4060, MLA4031, MLA4531, MFA2100, MLA3000							
Low hip circumference		cm (in)	65-71 (25.5-28)	72-79 (28.5-31)	80-85 (31.5-33.5)	86-93 (34-36.5)	94-105 (37-41.5)
		79-99 (31-39.5)	S	M	L	L	/
		100-116 (40-45.5)	M	M	L	XL	XXL
		117-133 (46-52.5)	M	L	L	XL	XXL
		134-154 (53-60.5)	L	XL	XL	XL	XXL
		155-170 (61-67)	/	XXL	XXL	XXL	XXL

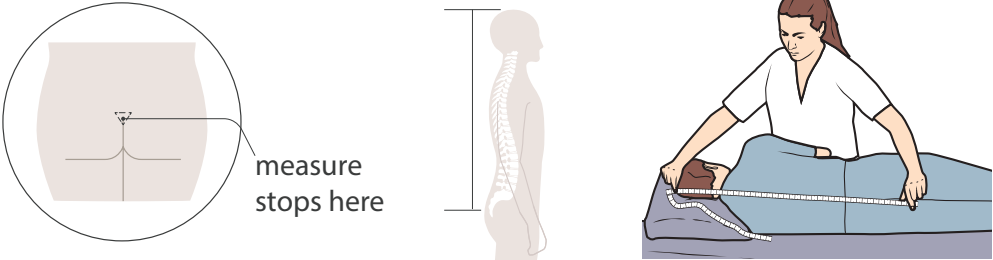
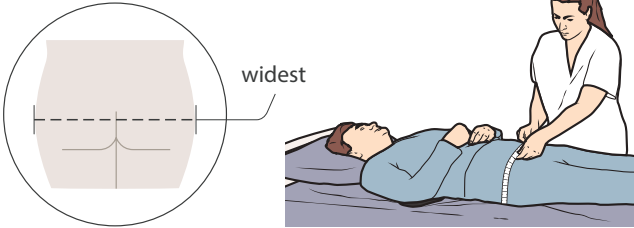
Pediatric sling

Sizing chart

Sling Model		Top of the head to coccyx				
MAA4020M						
		cm (in)	50 - 54 (19 1/2 - 21 1/4)	55 - 59 (21 1/2 - 23 1/4)	60 - 65 (23 1/2 - 25 1/2)	66 - 73 (25 3/4 - 28 3/4)
Low hip circumference 	50 - 59 (19 1/2 - 23 1/4)		XXS	XXS	XXS	XS
	60 - 69 (23 1/2 - 27 1/4)		XXS	XXS	XS	XS
	70 - 80 (27 1/2 - 31 1/2)		XXS	XXS	XS	S
	81 - 100 (31 3/4 - 39 1/2)		XXS	XS	S	S

Hammock slings

Sizing chart

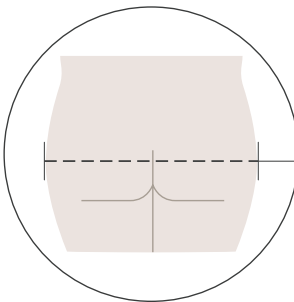
Sling Models		Top of the head to coccyx						
THA6i, THAi								
Low hip circumference		cm (in)	40-45 (16-17.5)	46-52 (18-20.5)	53-66 (21-26)	67-81 (26.5-32)	82-90 (32.5-35.5)	91-100 (36-40)
		55-70 (21.5-27.5)	XXS	XS	S	M	L	XL
		71-80 (28-31.5)	XS	XS	S	M	L	XL
		81-99 (37-41.5)	S	S	S	M	L	XL
		100-120 (39.5-47)	M	M	M	M	L	XL
		121-135 (47.5-53)	L	L	L	L	L	XL
		136-155 (53.5-61)	XL	XL	XL	XL	XL	XL

TIR sling

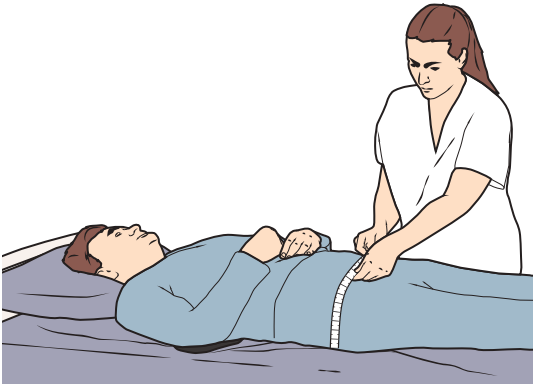
Sizing chart

Sling Model
TIR

Low hip circumference



widest



cm (in)		
85-119 (33.5-47)	120-139 (47-55)	140-166 (55-65.5)
S	M	L

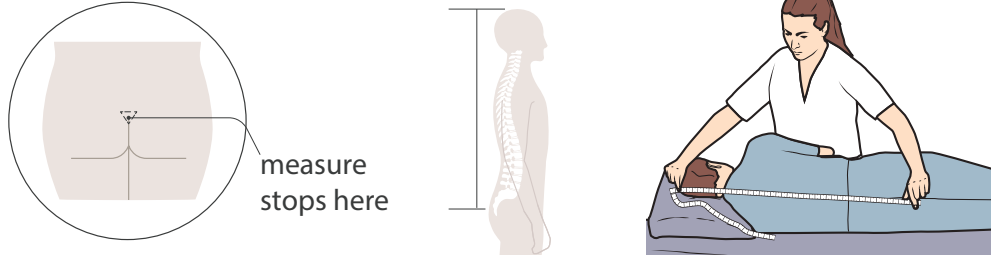
Double amputee clip slings

Sizing chart

Sling Models

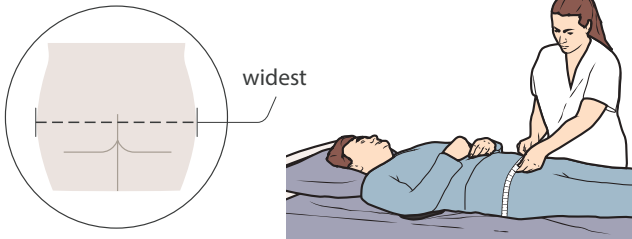
MAA2050M, MAA2080M,
MAA2090M, MAA2091M,
MAA4040M, MAA7090M

Measure top of the head to coccyx



cm (in)	60-65 (23.5-25.5)	66-73 (26-29)	74-81 (29-32)	82-89 (32.5-35)	90-95 (35.5-37.5)	96-103 (38-40.5)
70-80 (27.5-31.5)	XS	S	M	L	LL	XL
81-93 (32-36.5)	S	S	M	L	LL	XL
94-106 (37-41.5)	M	M	M	LL	LL	XL
107-125 (42-49)	M	L	L	LL	XL	XL
126-136 (49.5-53.5)	L	LL	LL	LL	XL	XL
137-155 (54-61)	XL	XL	XL	XL	XL	XXL
156-220 (61.5-86.5)	/	/	XXL	XXL	XXL	XXL

Low hip circumference



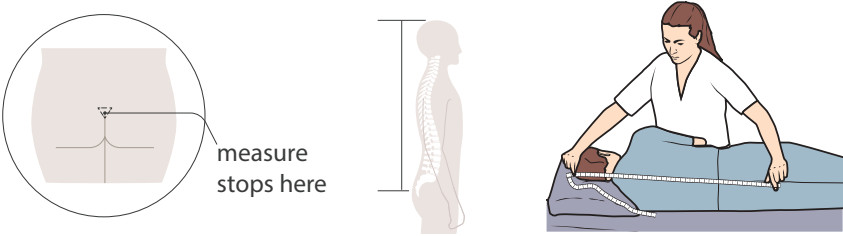
Single leg amputee clip slings

Sizing chart

Sling Models

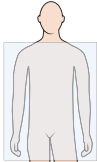
MAA4070M, MAA4080M

Top of the head to coccyx



measure stops here

Identify body type



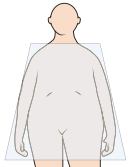
Rectangle body shape



Inverted trapezoid body shape

Or

Trapezoid body shape



Shoulder circumference



cm (in)	65-70 (25.5-27.5)	71-75 (28-29.5)	76-82 (30-32.5)	83-90 (32.5-35.5)	91-105 (36-41.5)
80-99 (31.5-39)	M	M	M	L	LL
100-114 (39.5-45)	M	M	L	LL	LL
115-130 (45.5-51)	L	L	L	LL	XL
131-149 (51.5-59)	LL	LL	LL	LL	XL
150-170 (59-67)	XL	XL	XL	XL	XL

Low hip circumference



cm (in)	65-70 (25.5-27.5)	71-75 (28-29.5)	76-82 (30-32.5)	83-90 (32.5-35.5)	91-105 (36-41.5)
80-95 (31.5-37.5)	M	M	M	L	LL
96-112 (38-44)	M	M	L	LL	LL
113-129 (44.5-51)	L	L	L	LL	XL
130-145 (51-57)	LL	LL	LL	LL	XL
146-166 (57.5-65.5)	XL	XL	XL	XL	XL

Hammock amputee sling

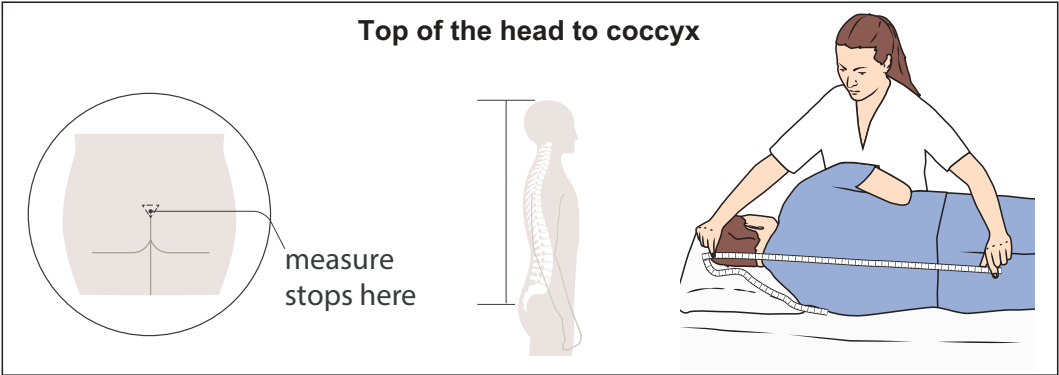
Sizing chart

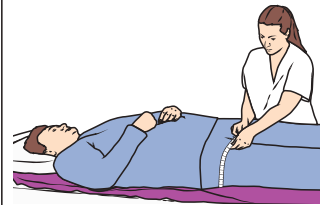
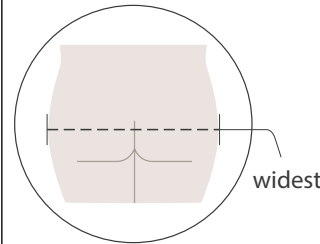
Sling Model
MLA7000

Shoulder circumference			
			
cm (in)			
80-120 (31.5-47)	105-125 (41.5-49)	110-130 (43.5-51)	120-140 (47-55)
S	M	L	XL

Bariatric slings

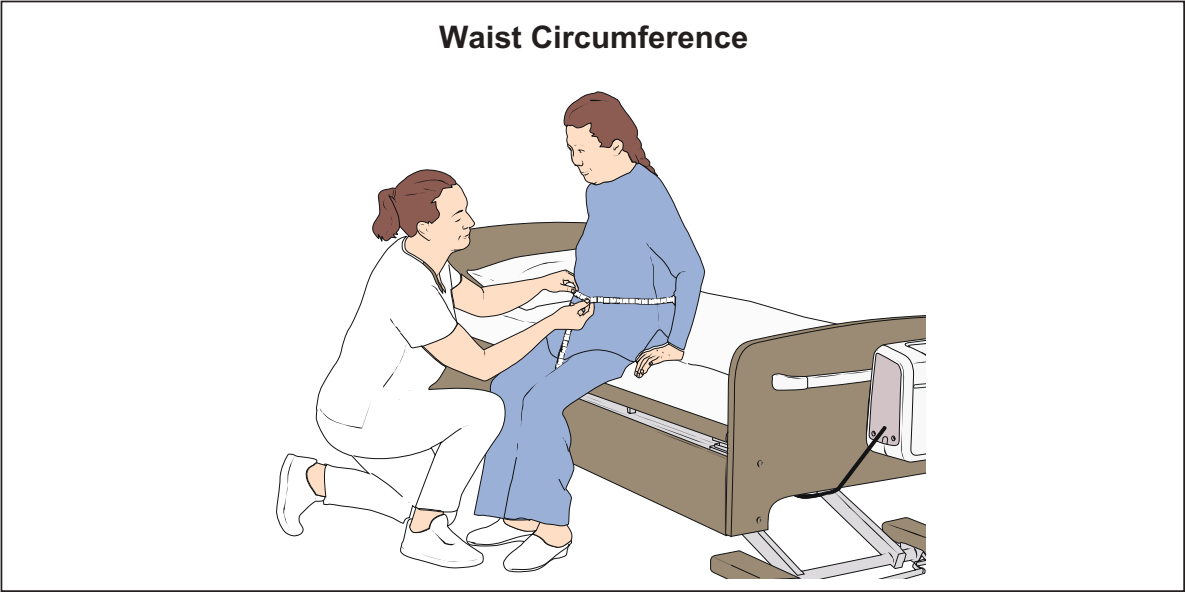
Sizing chart



Measure	Sling Models	cm (in)	72 - 79 (28 3/8 - 31 1/4)	80 - 85 (31 1/2 - 33 1/2)	86 - 93 (33 3/4 - 36 3/4)	94 - 105 (37 - 41 1/4)	106 - 115 (41 1/2 - 45 1/4)
Low hip circumference  	MAA8000 MAA8010 MFA8010	170 - 179 (66 7/8 - 70 1/2)	M	M	M	L	XL
		180 - 189 (70 3/4 - 74 1/2)	M	M	L	XL	XXL
		190 - 199 (74 3/4 - 78 1/2)	M	L	XL	XL	XXL
		200 - 209 (78 3/4 - 82 1/4)	L	XL	XL	XXL	XXL
		210 - 219 (82 1/2 - 86 1/4)	XL	XL	XL	XXL	/
	MAA8020 MAA8030	cm (in)	67 - 81 (26 3/8 - 32)	82 - 90 (32 1/4 - 35 1/2)	91 - 100 (35 3/4 - 39 1/2)	101 - 110 (39 3/4 - 43 1/4)	111 - 120 (43 1/2 - 47 1/4)
		155 - 164 (61 - 64 3/4)	M	L	XL	XL	XXL
		165 - 179 (65 - 70 1/2)	L	L	XL	XL	XXL
		180 - 194 (70 3/4 - 76 1/2)	L	L	XL	XL	XXL
		195 - 209 (76 3/4 - 82 1/4)	XL	XL	XL	XXL	XXL
		210 - 219 (82 1/2 - 86 1/4)	XL	XL	XXL	XXL	/

Standing transfer and walking slings

Sizing chart

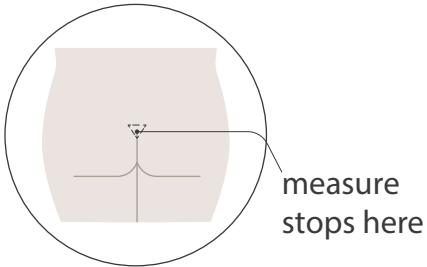
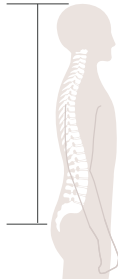
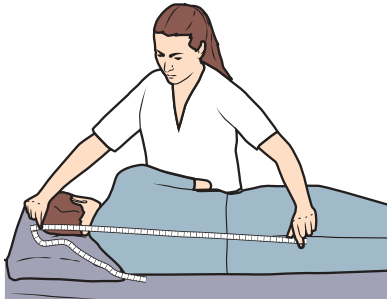
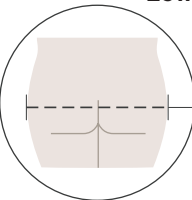


		S	M	L	XL	XXL	One size fits most
		cm (in)					
Sling Models	TSS.500-504	69-86 (27-34)	84-113 (33-44.5)	107-156 (42-61.5)	157-184 (62-72.5)	173-201 (68-79)	/
	TSS.500-503S-1C	63-130 (25-51)	89-130 (35-51)	91-133 (36-52.5)	104-160 (41-63)	/	/
	KKA5120, KKA5370, KKA5130M	70-97 (27.5-38)	98-117 (38.5-46)	118-137 (46.5-54)	138-169 (54.5-66.5)	/	/
	KKA6000, TSS.500W-TSS503W	69-86 (27-34)	84-110 (33-43.5)	100-135 (39.5-53)	120-160 (47-63)	/	/
	TEM	60-89 (23.5-35)	88-107 (34.5-42)	108-125 (42.5-49)	/	/	/
	MFA3000, MFA4000	/	/	/	/	/	80-125 (31.5-49)
	MAA3071	/	/	/	/	/	60-130 (23.5-51)

Loop walking sling

Sizing chart

Sling Model
MAA5000M

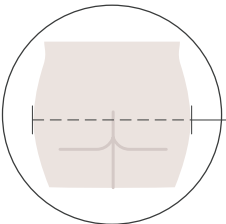
		Top of the head to coccyx			
					
cm (in)		65-74 (25.5-29)	75-83 (29.5-32.5)	84-89 (33-35)	90-94 (35.5-37)
	65-79 (25.5-32)	S	M	L	XL
	80-105 (31.5-41.5)	S	M	L	XL
	106-129 (41.5-51)	L	L	L	XL

Soft stretcher slings

Sizing chart

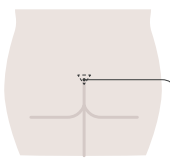
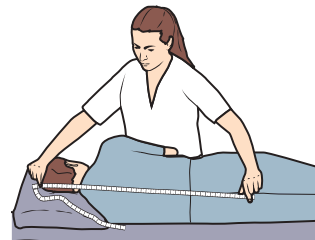
Sling Models	MAA1150, MAA1151, MAA1152
	MFA5000

Low hip circumference



widest



Top of the head to coccyx  measure stops here 	cm (in)	Maximum 130 (51)	Maximum 170 (67)
	Maximum 81 (32)	L	XL
	Maximum 90 (35.5)	/	One size fits most