

JAY[®]

LitePro[™]

JAY[®] LitePro[™] Cushion

IMPORTANT CONSUMER INFORMATION

NOTICE: *This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.*

SUPPLIER: *This manual must be given to the user of this product.*

USER: *Before using this product, read this entire manual and save for future reference.*

Owner's Manual

Cojín JAY[®] LitePro[™]

INFORMACIÓN IMPORTANTE PARA EL CONSUMIDOR

AVISO: *Este manual contiene instrucciones importantes que deben ser entregadas al usuario de este producto. Por favor no retire este manual antes de la entrega al usuario.*

DISTRIBUIDOR: *Este manual debe ser entregado al usuario de este producto.*

USUARIO: *Antes de usar este producto, lea este manual en su totalidad y guárdelo para futura referencia.*

Manual de
instrucciones

Coussin JAY[®] LitePro[™]

INFORMATIONS IMPORTANTES DESTINÉES AU CONSOMMATEUR

AVIS : *Ce manuel contient des instructions importantes qui doivent être communiquées à l'utilisateur de ce produit. Prière de ne pas le retirer avant livraison.*

FOURNISSEUR : *Ce manuel doit être remis à l'utilisateur de ce produit.*

UTILISATEUR : *Avant d'utiliser ce produit, lisez entièrement ce manuel et conservez-le pour référence ultérieure.*

Manuel d'
instructions



P/N SM100629 Rev. A

SUNRISE MEDICAL LISTENS

Thank you for choosing a JAY® product. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Sunrise Medical authorized dealer. Please feel free to write or call us

SUNRISE MEDICAL (US) LLC**Table of Contents**

General Warnings	3
Introduction.....	4
Cushion Adjustability.....	5
Maintenance & Cleaning.....	6
Sunrise Limited Warranty	7

Be sure to register your product and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products, and options to increase your use and enjoyment of this product.

You can register your product at:
www.SunriseMedical.com/register

FOR ANSWERS TO YOUR QUESTIONS

Your Sunrise Medical authorized dealer knows your product best, and can answer most of your questions about product safety, use, and maintenance. For future reference, please fill in the following:

Dealer: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this product was made with Natural Rubber Latex.

**DISPOSAL AND RECYCLING INFORMATION**

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This product is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your product properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations.



www.SunriseMedical.com/register

II. GENERAL WARNINGS

GENERAL WARNINGS

NOTE - Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Sunrise for further instructions.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY® LitePro™ cushion is appropriate. Cushions should only be installed by a Sunrise Medical authorized dealer.

WARNING

DO NOT install this equipment without first reading and understanding the manual. If you are unable to understand the Warnings, Cautions, and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.

WARNING

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue use of the cushion immediately and see your doctor or therapist.

The JAY LitePro Cushion is designed to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The LitePro Cushion is not a substitute for good skin care including, but not limited to: proper diet, cleanliness, and regular pressure reliefs.

WARNING

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

WARNING

Hook and Loop material should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Hook and Loop material.

WARNING

Allow the cushion to warm or cool to room temperature before using. Sitting on extremely hot or cold surfaces can cause skin damage.

WARNING

CAUTION - AVOID SHARP OBJECTS OR EXPOSURE TO EXCESSIVE HEAT OR OPEN FLAME.

WARNING

The pods contained in the insert are determined by the clinician during a seating evaluation. They should not be adjusted without a clinician directive as they can alter the pressure relieving properties and lead to injury.

WARNING

OBSTRUCTIONS: DO NOT place any obstructions between the user and the cushion as this will reduce product effectiveness.

WARNING

CUSHION AND COVER ORIENTATION: Product must be used with the insert oriented correctly. If the cover is not used correctly, it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY LitePro Cushion is appropriate. Cushions should only be installed by a Sunrise Medical authorized dealer.

JAY LitePro CUSHION

The JAY LitePro cushion is designed to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The JAY LitePro cushion is not a substitute for good skin care including proper diet, cleanliness, and regular pressure reliefs.

The JAY LitePro Cushion is designed for clinicians to use with clients who need superior pressure distribution and stability. The multizone elastomer insert has interchangeable pressure adjustment pods with differing levels of firmness to adjust the level of pressure management and accommodate pelvic obliquities. Pods should only be changed at the direction of your clinician.

Maximum user weight:

14-in to 20-in width (356mm to 508mm) = 330 lbs (150kg)

JAY LitePro COMPONENTS



IV. CUSHION ADJUSTABILITY

The JAY LitePro Cushion offers adjustability to accommodate for pelvic obliquity through interchangeable pressure adjustment pods with differing levels of firmness, which are located on the underside of the insert. The determination of the correct combination of pods should be made by your clinician during evaluation. Over time, as the user adapts, the pods may be removed or added to allow for maximum cushion to user contact. Pods should only be changed at the direction of your clinician.

WARNING

Place the insert in the base, making sure it is aligned properly in the grooves of the base. A misalignment or creation of an edge could cause an unwanted pressure point that may lead to injury

Fitting a Pressure Distributing Cushion

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1-in to 2-in (2.5 cm to 5 cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and insert toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischial tuberosities (sit bones) should be centered on the insert in the pelvic loading area of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the JAY Adjustable Solid Seat.

Use in freezing temperatures (less than 32° F / 0° C) may make the insert more firm and may affect its pressure-relieving qualities. If using the cushion in freezing temperatures, perform skin checks and pressure reliefs more often to ensure proper skin protection.

Exposure to heat and repeated pressure may make the insert compress slowly over time, compromising its pressure-relieving capabilities. Make sure to check your cushion monthly, or more often in warmer conditions, for signs of compression.

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.



Inspect

- Check the cover for tears and excessive wear and replace if ripped, torn, or otherwise not fully functional.
- Check the insert and pods for tears, deformation, or discoloration.
- Check the foam base to ensure foam consistency. Excessive wear and tear could compromise the performance of this cushion.
- If you notice any inconsistencies, contact your Sunrise Medical authorized dealer to inspect the components for possible replacement.

To Clean Cover

Cleaning Tips:

- If utilized in a multi-caregiver setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the cover after washing.
- Use only JAY designed covers on JAY cushions. The cover is an important part of allowing immersion into the insert which redistributes pressure. Changing the cover will alter this design and its efficacy.

Cleaning Steps:

1. Remove the cover from the foam base and turn it inside out.
2. Machine wash in warm water 60° C (140° F) and drip or tumble dry on low heat.
 - No ironing required.
3. To refit cover after washing, reinstall the cover on the foam base. Ensure the back of the cover matches up to the back of the foam base.

⚠ CAUTION

DO NOT DRY CLEAN COVERS OR USE INDUSTRIAL WASHERS AND DRYERS TO CLEAN COVER. DO NOT STEAM AUTOCLAVE. DO NOT BLEACH.

(Cushion Cover) Laundry Care Symbols			
	Machine wash in warm water (60° C)		Do not iron
	Only non-chlorine bleach, when needed		Do not dry clean
	Drip dry		

To Clean the Insert

Cleaning Tips:

- Normal household cleaners such as detergent, sanitizer, and bleach are all safe to use on the insert.
- Do not use strong alcohol-based cleaners, such as hydrogen peroxide, to clean the insert.
- Avoid exposing the insert to excessive oils such as lotions. Exposure to excessive amounts of strong acids and oils can make the elastomer material soften, compromising the pressure-relieving abilities of the insert.
- If you do use bleach on the insert, make sure it is completely rinsed and dried before putting it back together with the other components of the cushion as bleach will degrade the foam base and cover of this cushion.

Cleaning Steps:

1. Remove the cover from the foam base.
2. Remove the insert from the inner cover and handwash with warm water and soap, or machine wash, without any pressure on it. Let completely air dry before reassembly.
3. If removed, reinstall the pods into the insert.
4. Reinstall the insert on the base and reinstall the cover over the foam base.
5. Ensure the back of the cover matches up to the foam base.

To Clean Foam Base


1. Remove the cover and insert from the foam base and wipe lightly with a damp cloth.
 - Do not use soap.
 - Do not submerge in water.
2. Wipe off with a clean cloth.
3. Let completely air dry before reassembly.
4. Reinstall the insert on the base and reinstall the cover over the foam base.
5. Ensure the back of the cover matches up to the foam base.

If at any time, the cover, foam base, insert, or any other component appears to be wearing or you notice a degradation in the texture, contact your Sunrise Medical authorized dealer or Sunrise Medical customer service for evaluation and further instructions.

V. MAINTENANCE & CLEANING

Reassembling the JAY LitePro Cushion

1. Follow cleaning and maintenance instructions as described in "Maintenance & Cleaning" then reassemble as follows.
2. Place the insert in the foam base, lining up the contours to create a smooth transition between the insert and foam base (Fig 1).
3. If pods were removed for cleaning, reinstall them in the insert, making sure they are aligned properly and relatively flush with the insert (Fig. 2). If using an asymmetrical orientation (e.g. one firm pod, one soft pod), make sure to return the pods to the prescribed sides for proper pressure relief. The pods and insert can be marked with a permanent marker to ensure proper reassembly.

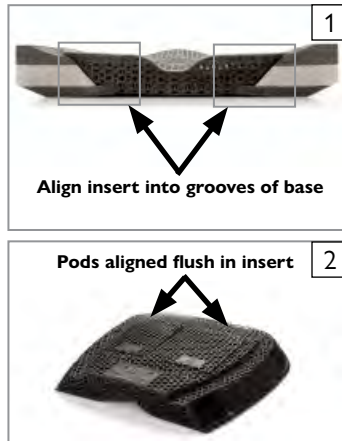

⚠ CAUTION

FAILURE TO LINE UP THE INSERT AND BASE ACCURATELY COULD RESULT IN PRESSURE INJURIES.

4. Insert the foam base into the cover. Ensure that the zipper is at the rear of the foam base where the pelvic loading area is located.

VII. SUNRISE LIMITED WARRANTY

JAY LITEPRO CUSHION WARRANTY

Each JAY LitePro Cushion is carefully inspected and tested to provide peak performance.

The foam base and multizone elastomer insert are guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

Claims and repairs should be processed through the nearest Sunrise Medical authorized dealer. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose, are excluded.

There are no warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

ADDITIONAL WARRANTY INFORMATION FOR SUNRISE MEDICAL PTY LTD IN AUSTRALIA ONLY:

For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to replacement or refund for a major failure and for compensation for any reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights and remedies under a law in relation to the goods to which the warranty relates.