GUARDIAN[®]

User's Guide

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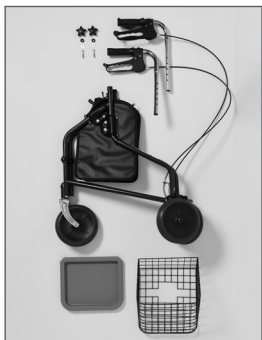
Warning

FAILURE TO FOLLOW THESE INSTRUCTIONS MAY RESULT IN INJURY.

- **DO NOT** use your walker until the handles are properly adjusted and securely tightened.
- Press folding mechanism past center line for full engagement prior to use.
- **DO NOT** use this walker on hills. A wheeled walker may not provide adequate support on slopes.
- **DO NOT** exceed the maximum weight capacity listed on the walker.
- During use, weight should be evenly distributed between the push handles.

Parts

- 2 - 5 point knobs
- 2 - washers
- 2 - black curved washers
- 2 - screws
- 1 - Basket
- 1 - Tray
- 1 - Frame with bag and left/right handles

Assembly

Install Handle Bars

1. Remove shipping caps from the frame's metal tube.
2. Insert Push handle into the frame (Figure 1).
3. Stand inside the walker and move the push handle up or down until it is in line with the bend of your wrist.
4. Insert 1 screw from the inside of the walker through the frame and tubing of the push handle at the desired height.
5. Add the black curved washer to the screw on the outside of the frame so the curve of the washer is against the frame.
6. Add the silver washer and the 5 point knobs and tighten (Hand Tighten only).
7. Repeat on other side. Make sure the push handles are set at the same height and fully tightened before use.

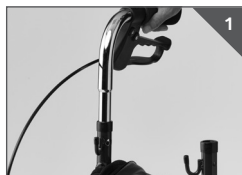
Figure 1

Using and Unfolding Your Walker

1. Push down on the center folding mechanism bar until the bar is past center line and fully engaged (Figure 2).
2. To fold, place all wheels on the ground. With one hand, pull-up on the center folding mechanism bar.

Figure 2

Locking and Unlocking Your Walker's Brakes

1. To lock brakes, push down on brake handle until it "clicks" into the locked position (Figure 3).
2. To unlock, pull up on brake handle.

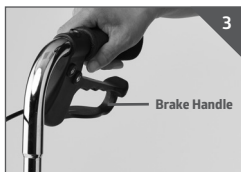
Figure 3

Safety Instructions

- **DO NOT** use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technical personnel before use - otherwise serious bodily injury and damage may occur.
- Consult with your doctor, physical, or occupational therapist or other qualified professional to assure the proper height adjustment.
- Serious injury to you and/or damage to the rollator walker frame or wheels may result from improper use.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result: **Maximum weight capacity: 275 lb (124 kg).**
- **DO NOT** exceed maximum weight capacity on the tray. Maximum capacity is 10 lb (4.5 kg).
- Before each use make sure the walker frame is fully opened and locked.
- Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order. Wheels should roll and not wobble.
- All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- **DO NOT** perform any adjustments to the rollator while it is in use.
- Only use accessories and spare parts authorized by Medline Industries, LP.
- **DO NOT** push rollator while brakes are enabled. Brakes are used to keep the wheels from rolling. But it may not prevent the rollator from moving if it is pushed.
- **DO NOT** use the rollator to provide assistance while rising from a lower position. If you use it to pull yourself up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- Inspect the handle bar grips prior to use. Assure the plastic parts are still functional if they are exposed to extreme temperatures (above 100°F (38°C) or below 32°F (0°C) high humidity and/or become wet.
- **DO NOT** twist hand grips as they are not meant to be twisted. Failure to do this could result in a fall.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2022 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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