NL

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MIC-KEY® BOLUS EXTENSION SET

Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Single Patient Use Only.

Warning: For enteral nutrition and/or medication only.

 Nonsterile	Not made with natural rubber latex	Product is NOT made with DEHP as a plasticizer	Rx Only
 Caution	 Consult instructions for use	 2797	

Distributed in the USA by Avanos Medical Sales, LLC,5405 Windward Parkway, Alpharetta, GA 30004 USA. In USA, 1-844-4AVANOS (1-844-428-2667) . www.avanos.com
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Avanos Medical Belgium BVBA, Leonardo Da Vincilaan 1, 1930 Zaventem, Belgium.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067 Australia.
*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2019-01-30

MIC-KEY® Bolus Extension Set with Cath Tip, SECUR-LOK® Right Angle Connector, and Clamp

For MIC-KEY® Low-Profile Gastrostomy Tubes

Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

1. Remove the plug from the MIC-KEY® feeding tube to reveal the MIC-KEY® feeding tube anti-reflux valve.

2. For gastric decompression or enteral feeding, connect the Extension Set connector by aligning the black line on the connector with the black line on the MIC-KEY®. Lock into place by rotating it CLOCKWISE 3/4 turn.

3. Flush the Extension Set and the MIC-KEY® with warm water (10 ml for adults, 3–5 ml for children). Check for ease of flushing.

4. After feeding or decompression, flush the Extension Set and the MIC-KEY® with 5–10 ml of warm water.

5. To disconnect the Extension Set from the MIC-KEY® rotate the extension set connector COUNTER-CLOCKWISE until the black line on the MIC-KEY® aligns with black line on the Extension Set. Gently remove the connector from the MIC-KEY®. Replace the plug into the MIC-KEY®.

6. After each use, wash the Extension Set with warm, soapy water and rinse thoroughly.

Single Patient Use Only.

Not made with natural rubber latex.

Product is not made with DEHP as a plasticizer.

Warning: For enteral nutrition and/or medication only.

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