VITÁLE®

ADHESIVE FOAM DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Adhesive Foam Dressings are highly absorbent and provide an ideal moist environment for healing. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time of up to seven days. These dressings are indicated for the treatment of moderate to heavy exudating wounds, and assist in protecting the wound, preventing bacterial contamination and maceration.

There are two layers, an absorbent foam that absorbs and retains exudates, and a waterproof yet vapor permeable film to provide protection. They feature a waterproof film border with non-allergenic adhesive to adhere the dressing in place. The breathable film also allows excess gases to escape to allow moisture absorption.

Vitále® Adhesive Foam Dressings are sterilized by radiation and must not be resterilized.

INDICATIONS

Vitále® Adhesive Foam Dressings are for single use only and can be used for the treatment of the following wounds:

- Pressure Ulcers
- Arterial Ulcers
- Venous Ulcers
- Skin donor site
- Deep & Superficial Wounds
- Moderate to Heavy Exudating Wounds
- Diabetic Ulcers
- Second Degree Burns

CAUTION

Do not use with sodium hypochlorite or hydrogen peroxide solutions.

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitále® Adhesive Foam Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean and rinse the wound, using saline or appropriate wound cleanser.
2. Remove the release paper exposing the patient contact layer, using sterile technique.
3. Place the Vitále® Adhesive Foam Dressing on the moist wound. The foam contact layer should cover the wound bed. If the adhesive version is used, the border should be adhered to peri-wound tissue. Do not stretch the dressing while applying.
4. Change dressing as directed, when the foam is saturated, or at least every 7 days.

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