HYDROGEL IMPREGNATED DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Hydrogel Impregnated Dressings feature a soft gel impregnated into an absorbent dressing designed for direct wound contact. Hydrogel provides a moist wound environment which supports the body's healing process & promotes autolytic debridement (removal of necrotic material from the wound) without damaging new tissue. The cotton dressing that holds the hydrogel allows removal of the entire dressing intact at dressing change time.

Vitále® Hydrogel Dressings are a primary dressing, and are most often used on wounds that are dry to moderate drainage. They should be covered with an occlusive or semi-occlusive secondary dressing that will maintain the moisture in the dressing & wound. Vitále® Hydrogel Impregnated Dressings are sterilized by irradiation and must not be re-sterilized; the products are sterile unless the package is opened or damaged. Vitále® Hydrogel Dressings are for single use only.

INDICATIONS

Vitále® Hydrogel Impregnated Dressings can be use for wounds which are:

Shallow or Deep • Pressure Sores
• Arterial Ulcers
• Venous Ulcers

Low to Moderate • Diabetic Ulcers
Drainage • Post-Operative Wounds
• Wounds that need autolysis

CAUTION

- Do not use in cases of known sensitivity to Vitále® Hydrogel Dressings or any of their components.
- Vitále® Hydrogel Dressings should be used under the supervision of a health care professional.

- Vitále® Hydrogel Dressings may be used on infected wounds under medical supervision and with appropriate therapy & frequent monitoring.

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitále® Hydrogel Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean the wound carefully per existing clinical protocol. The skin surrounding the wound should be clean and dry.
2. Select a dressing that fits into the wound. If necessary, fold to adjust to wound size. Do not extend the dressing over the peri-wound tissue, to prevent maceration.
3. Carefully remove the dressing from the sterile package, applying the center of the dressing to the center of the wound, using clean technique. Press gently, forming the dressing into the contours of the wound.
4. Apply an occlusive or semi-occlusive secondary dressing.
5. Change daily, or more often, as needed. Change when the dressing is dry to keep a moist dressing in place. Change if leakage of exudate occurs through the secondary dressing.
6. If Vitále® Hydrogel Dressings are used in the presence of necrotic tissue, the wound may appear to increase in size and depth during autolysis. This is a normal consequence of occlusive treatment. If the wound continues to get larger after the first few dressing changes, consult a healthcare professional.
7. Rinse before wound assessment and applying a new dressing.

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