



Attainment's  
GoNow accessories  
help to enhance  
and protect your  
iPad when used  
as an AAC device,  
ensuring that  
communication  
ability is maximized  
for every voice.

[www.AttainmentCompany.com](http://www.AttainmentCompany.com)



## GoNow™ Accessories

### Screen Protector

The Screen Protector helps protect your iPad screen from moisture and smudge marks. The vinyl sheet fits snugly in both Sleek and Rugged models. The Protector is packaged in a sealed envelope to prevent exposure to dust or other fine particles. Available for all Mini and iPad models.



1 Remove the plastic iPad screen protector from the envelope.



2 Place the screen protector into the front piece of the GoNow Case.



3 Place the iPad, screen down, into the front piece of the GoNow Case. Align the iPad charging dock with the Case access port.



4 Attach the back of the Case and tighten all the screws.

## CoverStand

The reversible CoverStand functions as both an adjustable stand and a durable cover to protect the iPad's glass screen in transport or storage. As a stand, it snaps on the back of the GoNow Case. Pull out the brace and adjust the viewing angle. It snaps on the front to completely cover the screen. *Note: Do not try to snap the CoverStand on a GoNow Case unless the screws are fully tightened.* The CoverStand works with GoNow Sleek and Rugged Cases for iPads 2, 3, and 4. It does not fit the Mini models.



**1 STAND** Place the GoNow Case into the bottom groove of the CoverStand with the iPad screen facing out.



**2** Squeeze the top groove of the CoverStand over the top of the case.



**3** Pull out the brace on the back.



**4** Set the Case and CoverStand on the table to view the iPad at a 45° angle.



**5** Push in the brace when not in use.



**6** Push out on the top groove of the CoverStand to remove it from the Case.



**7 COVER** Place the GoNow Case into the bottom groove of the CoverStand with the iPad screen facing the inside.



**8** Squeeze the top groove of the CoverStand over the top of the Case.



**9** Push out on the top groove of the CoverStand to remove it.



## Strap

The Strap fits all iPad and Mini GoNow Cases. The durable clip-locks open and close easily. The length can be adjusted to fit both a lanky teenager and a petite first grader.

1



To connect the Strap to the GoNow Case, feed the end into the lanyard holes on the GoNow Case.

2



Open the plastic clip-lock and feed in the end of the Strap.

3



Flip down the clip-lock to grip the Strap in place.

4



To adjust the Strap length, slide the pad to the left to reveal the adjustable Velcro® Strap.

5



Pull apart the Velcro®.

6



Press the Velcro® together at the desired length.

7



Slide the pad back over the Velcro®.

***Works with all GoNow Cases  
and adjusts to fit all students  
regardless of height!***

# Turn your iPad into a GoTalk!


Attainment's



A full-featured, customizable AAC tool ideal for the beginning or experienced communicator



Scan this code with your iPad!

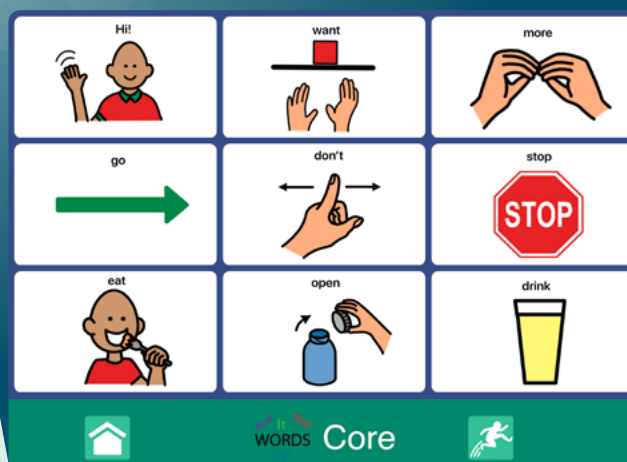

**GoNow Cases**  
for older iPads  
still available



**Attainment  
Company**