


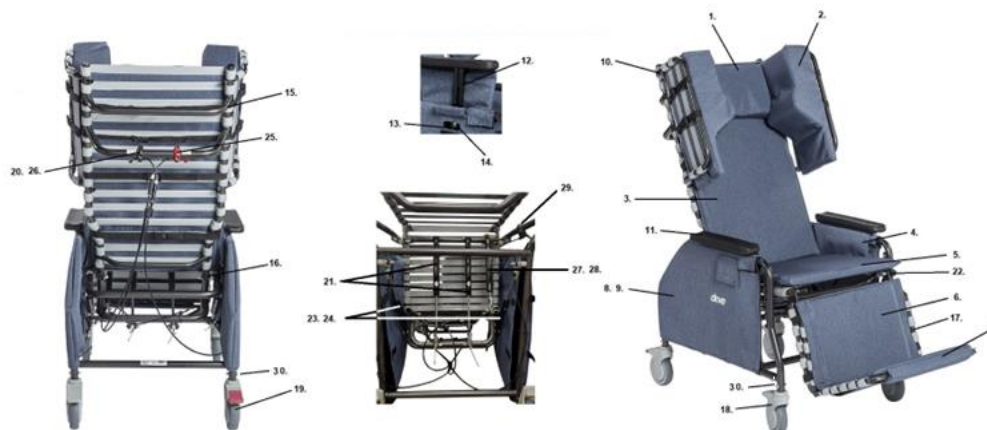

- | | |
|---|---|
| 1. Center Headrest Cushion- CLN400-20-01 | 14. Armrest Height Adj. Assy.- CLN400-20-13 |
| 2. Side Headrest Cushion- CLN400-20-02 | 15. Back Frame- CLN400-20-14 |
| 3. Backrest Cushion- CLN400-20-03 | 16. Backrest Attaching Hardware- CLN400-20-15 |
| 4. Clothing Guard Cushion- CLN400-20-04 | 17. Leg/Footrest Assembly- CLN400-20-16 |
| 5. Seat Cushion- CLN400-20-05 | 18. 5" Directional Caster- CLN400-20-29 |
| 6. Calf Pad- CLN400-20-06 | 19. 5" Locking Swivel Wheel- CLN400-20-18 |
| 7. Foot Pad- CLN400-20-07 | 20. Tilt Lever Hardware Set- CLN400-20-19 |
| 8. Sideframe Cover Left- CLN400-20-08L | 21. Tilt Piston- CLN400-20-20 |
| 9. Sideframe Cover Right- CLN400-20-08R | 22. Seat Frame Hardware- CLN400-20-21 |
| 10. Side Headrest Fixed Frame- CLN400-20-09 | 23. Reclining Piston Hardware-- CLN400-20-22 |
| Side Headrest Adj Frame Left- CLN400-20-09LN | 24. Reclining Piston- CLN400-20-23 |
| Side Headrest Adj Frame Right- CLN400-20-09RN | 25. Reclining Lever & Cable- CLN400-20-24 |
| 11. Arm Pad with Hardware- CLN400-20-10 | 26. Tilt Lever & Cable- CLN400-20-25 |
| 12. Upper Arm Assembly Left- CLN400-20-11L | 27. Leg Piston- CLN400-20-26 |
| Upper Arm Assembly Right- CLN400-20-11R | 28. Leg Piston Hardware- CLN400-20-27 |
| 13. Armrest Receiver & Hdwr Left- CLN400-20-12L | 29. Leg Lever & Cable- CLN400-20-28 |
| Armrest Receiver & Hdwr Right- CLN400-20-12R | 30. Caster C Clips (4 ea)- CLN400-200-30 |

04/07/2025