GUARDIAN®

Family-owned. Trusted by hospitals for over 50 years.
Entreprise familiale. Marque approuvée par les hôpitaux depuis plus de 50 ans.
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

LIFETIME
LIMITED
WARRANTY

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ECONOMY BARIATRIC SHOWER CHAIR

USER INSTRUCTIONS AND WARRANTY

Weight capacity 550 lbs. (249 kg)

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REF G2-102BX1

CAUTIONS

1. Inspect all parts for possible damage before assembling. If there is any damage - **DO NOT USE**. Contact dealer/carrier for further instructions. Allen hex tool and wrench included for assembly.
2. Make sure that each U-Clip extends fully into the same corresponding hole of each leg extension. This ensures that the leg extensions are securely locked in place at an even height.
3. Make sure all screws, nuts and/or bolts are tightened.
4. Make sure the rubber tips are in place on the leg extensions and not ripped, worn or missing. Replace any missing or damaged tips.
5. The rubber tips of all four leg extensions **MUST** touch the floor simultaneously at all times.
6. NEVER stand on chair.

PARTS DIAGRAM

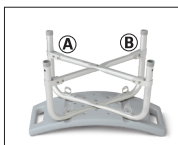
Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

Figure 6

Figure 7

Figure 8

ASSEMBLY AND USE

READ ALL INSTRUCTIONS CAREFULLY BEFORE USING THIS CHAIR.

1. Place seat upside down on work surface.
2. Place cross bar B into diagonal seat channel and cross bar A perpendicular over B in other seat channel. **(Figure 1)**
3. Use the Allen hex tool to attach the four screws securing the seat to the frame. **(Figure 2)**
4. Secure middle screw with washer into the middle position on frame and seat. **(Figure 3)**
5. Secure middle screw on frame using Allen hex tool and wrench. **(Figure 4)**
6. Slide first leg into frame, adjust to height, then insert C-Clip into pre-drilled holes. Repeat for each leg at same height. **(Figure 5)**
7. Begin assembling the backrest by placing it frontside down on the assembled seat. Line up each backrest tube and secure to the seat using 2 screws with washers on each tube. **(Figure 6)**
8. Lie the assembled seat on the floor with the round tube brackets facing up. Slide the assembled backrest tubes into the tube brackets **(Figure 7)** and secure using 2 C-Clips in holes closest to the back of the seat. **(Figure 8)** Stand up seat, make sure all fasteners are secure, and adjust leg height if needed.

ADJUSTING LEG HEIGHT

1. To adjust leg height, remove C-Clip from adjustment hole, set leg to desired height and return U-clip into adjustment hole. Make sure C-Clip clicks securely into place.
2. Repeat for remaining legs making sure all are locked into place at the same height.
3. Push down and pull up on each leg to be sure legs are locked into place.

Before using the seat, make sure all legs are set to the same preferred height.

CAUTION: Maximum weight limit for this chair is 550 lbs (249 kg).

MAINTENANCE

Periodic visual inspection of this bath chair is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE

NOTE: To maintain appearance and promote good hygiene, clean periodically with mild soap and rinse with warm water. Wipe bath chair dry after each use.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN®

Medline Industries, LP,
Three Lakes Drive
Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
info@medline.com

Medline Canada
1-800-268-2848
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2020 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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