




CarePocket[®] by Tytex

Leg bag holder

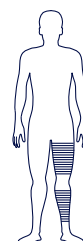

CarePocket[®] is specially designed to fixate a urine leg bag, without impeding appearance. The weight of a full leg bag is supported by the whole product. This way, the tension required to hold a bag in place is distributed evenly around the leg, rather than being concentrated in two narrow rings at the top and the bottom of the bag.

Features:

- Easy to use
- Safe retention of leg bag for optimum support
- Tension is distributed evenly around the leg
- Provides discretion
- Very soft, stretchy and breathable material, that is comfortable to wear
- Fits left and right leg equally well
- Fits both thigh and calf
- Bottom tap opening for easy emptying
- Able to contain urine leg bags of volumes up to 750ml/24/25oz
- It is recommended to empty the urine leg bag when 1/2 full
- Practical colour code for size indication
- Washable at 60°C/140°F
- OEKO-TEX certified

Use:

CarePocket[®] features a pocket within the design for the urine bag. At the bottom there is a small hole, and the valve of urine bags with bottom tap opening can be pulled through.

Composition:

CarePocket[®]: 80% Polyester, 20% Elastane.

Washing instructions:



Sizes available:

CarePocket[®] S-XL.

Instruction for use:



1. Pull on the leg bag holder (ensure the color size code is at the top).



2. Place the leg bag in the pocket and pull the tap through the lower hole.



3. The CarePocket[®] is in place.



ISO 9001
ISO 13485
BUREAU VERITAS
Certification



ISO 14001
BUREAU VERITAS
Certification



Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485