
## MIC-KEY® MEDICATION EXTENSION SET

### Instructions for Use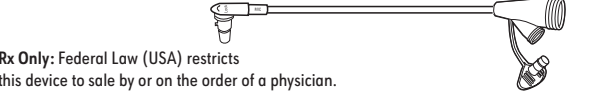

**Rx Only:** Federal Law (USA) restricts this device to sale by or on the order of a physician.

**Single Patient Use Only.**

**⚠ Warning: For enteral nutrition and/or medication only.**

|  |            |                                    |                                                |         |
|--|------------|------------------------------------|------------------------------------------------|---------|
|  | Nonsterile | Not made with natural rubber latex | Product is NOT made with DEHP as a plasticizer | Rx Only |
|  | Caution    |                                    | Consult instructions for use                   |         |

Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA. In USA, 1-844-4AVANOS (1-844-428-2667) . [www.avanos.com](http://www.avanos.com)  
 Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.  
 Avanos Medical Belgium BVBA, Leonardo Da Vinciiaan 1, 1930 Zaventem, Belgium.  
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067 Australia.  
\*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.  
© 2018 AVNS. All rights reserved. 2019-01-31

## MIC-KEY® Medication Extension Set with SECUR-LOK® Right Angle Connector and 2 Port “Y”

### Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

- Remove the plug from the MIC-KEY® feeding tube to reveal the MIC-KEY® feeding tube anti-reflux valve.
- Snap the Medication Set into place by aligning the black line on the Medication Set with the black line on the MIC-KEY®. Lock into place by rotating it CLOCKWISE 3/4 turn.
- Flush the Medication Set and the MIC-KEY® with 1 ml to 2 ml of warm water. Check for ease of flushing before medication administration.
- After medication administration, flush the Medication Set and the MIC-KEY® with 1 ml to 2 ml of warm water.
- To disconnect the Medication Set from the MIC-KEY® rotate the Medication Set COUNTER-CLOCKWISE until the black lines on the Medication Set and the MIC-KEY® are aligned. Gently remove the Medication Set from the MIC-KEY®. Replace the plug into the MIC-KEY®.

- After each use, wash the Medication Set with warm, soapy water and rinse thoroughly.
- Single Patient Use Only.**  
Not made with natural rubber latex.  
Product is not made with DEHP as a plasticizer.
- ⚠ Warning: For enteral nutrition and/or medication only.**

REF 0122-02

⚠ Warning: For enteral nutrition and/or medication only.

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