en Male Instructions

To prepare
Wash hands. Use mild, unscented soap or a non-alcoholic wet wipe to cleanse glans and opening of urethra as instructed by your healthcare practitioner. Choose a position to catheterize that is most comfortable for you (standing, sitting on the toilet, in wheelchair).

Using finger holes peel open the package until the funnel and gripper are visible. Hold funnel and/or gripper to remove catheter.

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
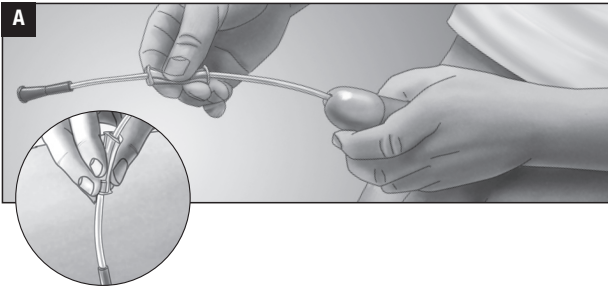
To use
Pull back foreskin if present.
A : Pull gripper away from the funnel. Hold gripper anywhere along the catheter, so you do not touch the catheter. With the penis stretched and in upright position, hold gripper tightly to begin inserting catheter. Slightly release gripper, and slide it back towards funnel. Then hold gripper tightly again to advance the catheter. When using a Tiemann /Coudé tip catheter, locate the notch on the funnel, which is aligned with the upward direction of the Tiemann /Coudé tip. Insert the catheter tip into the urethra with the notch in the 12 o'clock position, unless instructed otherwise by your healthcare practitioner. Repeat until urine starts to flow. Relax foreskin, if present.

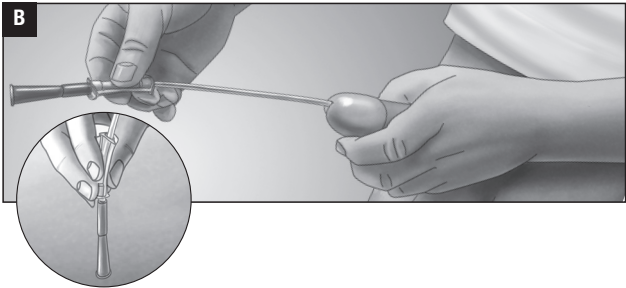B : Alternative insertion option: keep the gripper 'docked' with the funnel. With penis stretched and in upright position, hold gripper/funnel and insert catheter into the urethra with one hand. When using a Tiemann /Coudé tip catheter, locate the notch on the funnel, which is aligned with the upward direction of the Tiemann /Coudé tip. Insert the catheter tip into the urethra with the notch in the 12 o'clock position, unless instructed otherwise by your healthcare practitioner. Advance catheter until urine starts to flow Relax foreskin, if present.

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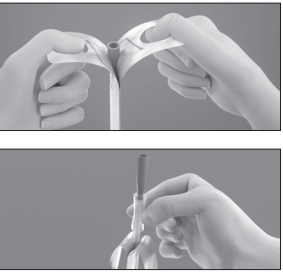
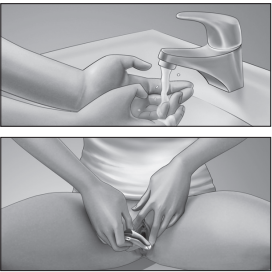
en Female Instructions

To prepare
Wash hands. Spread the labia and cleanse around opening of urethra with mild, unscented soap or a non-alcoholic wet wipe as instructed by your healthcare practitioner. Choose a position to catheterize that is most comfortable for you (standing, sitting on the toilet, in wheelchair or lying in bed).

Using finger holes peel open the package until the funnel is visible. Hold funnel to remove catheter.

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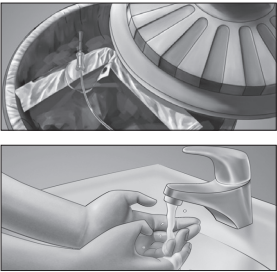
To remove and dispose (Please follow all local disposal regulations)
Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Catheter may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash hands.

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