
AVANOS

NEOMED[®]
CLEANING TOOL
ENFit[®] COMPATIBLE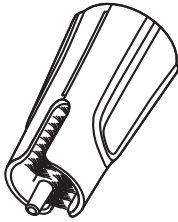

- Appareil de nettoyage, compatible avec les seringues ENFit[®]
- Herramienta de limpieza compatible con ENFit[®]
- Ferramenta de limpeza compatível com ENFit[®]

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Instructions for Use

Notice: This device is not intended to be used as a standalone device and can only be used with Feeding Tubes and Extension Sets with an ENFit[®] male connector. Read this complete use guide and the Feeding Tubes and/or Extension Sets with an ENFit[®] male connector (as applicable) Instructions for Use (IFU) prior to using this device. For additional warnings and precautions, please see IFUs of Feeding Tubes and/or Extension Sets with an ENFit[®] connector.

Device Description / Intended Purpose/ Indications for Use

NEOMED[®] Cleaning Tool is used to remove debris from a NEOMED[®] Feeding Tube or Extension Set with an ENFit[®] male connector for Neonates and Pediatrics.

Intended Users

This device is intended to be used by healthcare professionals.

Patient Target Group

This device is intended for use in neonatal and pediatric patients.

Residual Risks and Side Effects

While side effects vary depending on the type of procedure performed, mitigations have been implemented to minimize the following residual risks related to the use of the cleaning tool: allergic/foreign body/toxic reaction, infection.

Warnings/Cautions

Do not reuse devices labeled for single use only. Reuse of a single use device that has come in contact with blood, bone, tissue or other body fluids may lead to patient or user injury. Possible risks associated with reuse of a single use device include, but are not limited to, mechanical failure and transmission of infectious agents.

Do not use if primary pouch has been previously opened or damaged as this may lead to device contamination or biological infection.

Do not autoclave or re-use the NEOMED[®] Cleaning Tool as this may increase the risk of biological infection. Ensure the plug stays engaged during use.

Medical device	Single Use Only	Sterilized using ethylene oxide	Single sterile barrier system
Single sterile barrier system with protective packaging outside	Do not use if package is damaged	Do not resternalize	
MR Unsafe	Caution	Consult instructions for use	Keep dry
Keep away from sunlight	Country of manufacture	Manufacturer	Catalogue number
Unique Device Identifier	Batch code	Date of Manufacture	Use-by date

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MR Information

MR Unsafe

Cleaning tool is MR unsafe. Cleaning tool is intended for transient use in a non-MR imaging environment. It is not reasonably foreseeable that the device would be used in the presence of MR imaging environments.

Device Lifetime

Devices are validated for use up to 24 hours.

Complaint and Adverse Event Reporting

If a serious incident has occurred in relation to the device, it should be reported to the manufacturer and the competent authority in which the user and/or patient is established. A serious incident means any incident that directly or indirectly led, might have led or, in case of recurrence, could lead to any of the following: the death of a patient, user or other person, the temporary or permanent serious deterioration of a patient's, user's, fetus or other person's state of health, or a serious public health threat.

Disposal

Contact the local authorities to determine the proper method of disposal of potentially biohazardous parts and accessories. Disposal of the device and accessories should be in accordance with local, state, and national laws and regulations.

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*Registered Trademark or Trademark of
© 2018 AVNS. All rights reserved. 2024-03-04