XEROFORM PETROLATUM DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Xeroform Petrolatum Dressings are non-adherent occlusive dressings made of fine mesh gauze impregnated with 3% Xeroform (Bismuth Tribromophenate) in petrolatum. Xeroform Dressings are sterile, single-use devices.

INDICATIONS

Xeroform Petrolatum Dressings are intended for use as a primary contact layer to dress wounds such as lacerations, skin graft sites, newly sutured wounds, abrasions and minor or partial thickness burns. They may also be used as an initial layer in dressing pressure wounds with light exudate.

**Superficial to
Partial Thickness**

- Incision Sites
- Donor Sites & Skin Graft Sites
- Abrasions & Minor Burns
- A Primary Occlusive Layer

**Dry to Light
Exuding**

CAUTION

1. Do not use on patients with hypersensitivity to Bismuth Tribromophenate.
2. Caution should be used when there is a history of atopy.
3. Do not use on heavily draining wounds.

4. Do not use if unit package is opened or damaged.
5. Avoid multiple layers which may lead to maceration.
6. Discontinue use if irritation develops.
7. Avoid excessive packing and resultant pressure on the wound.

SIDE EFFECTS & INTERACTIONS

Irritation, though rare, may develop in atopic patients or in patients sensitive to Bismuth Tribromophenate.

STORAGE

Vitále® Xeroform Petrolatum Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean and rinse the wound, using appropriate wound cleanser.
2. Before applying, skin around the wound should be clean and dry. Use a skin barrier prep as needed.
3. Using aseptic technique, remove dressing from the package and apply to moist wound bed.
4. Cover with a secondary absorbent dressing, and secure as appropriate.
5. Dressing change as directed, when strike-through occurs, or at least every 7 days.

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