MIC*

Verlengslang

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
MIC* EXTENSION TUBING

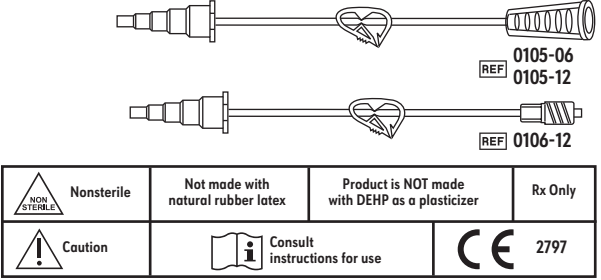Instructions for Use

Distributed in the USA by Avanos Medical Sales, LLC,5405 Windward Parkway, Alpharetta, GA 30004 USA. In USA, 1-844-4AVANOS (1-844-428-2667) . www.avanos.com
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
EC REP Avanos Medical Belgium BVBA, Leonardo Da Vinciloan 1, 1930 Zaventem, Belgium.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067 Australia.
*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2019-01-25

MIC* Extension Tubing

Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

- Attach a feeding syringe or set to the MIC* Extension Set and flush.
- Insert the MIC* Extension Set connector into the open feeding port. Use a FIRM twist to make a secure connection. Begin feeding. After feeding, flush the MIC* Extension Set with 5–10 ml of warm water. During continuous feeding, flush approximately every 4 hours.
- Disconnect the feeding set from the feeding port with a FIRM twist.
Note: Always hold the universal connector firmly when handling the MIC* Gastrostomy Tube.

1. Irrigate the MIC* Gastrostomy Tube with warm water using 10 ml for children or 20 ml for adults, injected through the feeding port.

5. Close the feeding port securely.

Single patient use only.

Not made with natural rubber latex.

Product is not made with DEHP as a plasticizer.

⚠ Warning: For Enteral Nutrition and/or Medication Only.

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