



# Tricycle

Adaptive tricycles give non-ambulatory people the joy and freedom of independent movement. **The Rifton Tricycle** combines four decades of expertise with therapists' insights and cutting-edge technology to create the ultimate ride. It's sleek, rugged, adaptable, and a whole lot of fun.



Scan here to order, request a quote, or see detailed product information.

- Accessible and functional design
- Tool-free adjustments
- Therapeutic benefits including improved leg strength, posture, balance, and range of motion





# Key features

## 1 Seat adjustability

The seat glides smoothly up and down on a low-friction rail with height markings in increments.



## 2 Durable materials

Built to last with premium materials: carbon fiber drive belts, high traction solid urethane tires, and automotive-grade powder-coated frame finish.



## The Tricycle in use



### 3 Variable resistance and parking brake

The adjustable brake can provide resistance or lock the wheels for safe transfer.



### 4 Freewheel option

The freewheel option allows caregivers to disconnect the pedals from the drivetrain so the tricycle can be pushed by the caregiver while the pedals remain stationary.





# Key features

## 5 Backrest adjustments

Position the backrest vertically or at a 7½° or 15° recline.



## 6 Easy transport

Remove rear wheels and front assembly, then flip down the backrest for compact transport.



## The Tricycle in use

---



# Tricycle options



Scan for sizing  
and pricing.

## Frame size options



Large



Medium



Small

## Frame color options

### Small and medium Tricycle colors



Blue



Red



Teal



Lime



Pink

### Large Tricycle colors



Blue



Red



Teal



White



Black

## Handlebar options



### Handlebars

Choose between stylish conventional handlebars, or loop handlebars for more adjustability.

## Backrest options



### Trunk support

Stabilizes riders with low trunk control. Includes butterfly harness for front support, with optional lateral supports and headrest.



### Backrest pad

For clients who need less support.

# Tricycle components



## Seats

Soft foam for comfort. Includes a seat belt.

**Small:** 8" wide x 8" long

**Medium:** 13" wide x 10" long

**Large:** 17" wide x 12" long



## Headrests

Winged or contoured headrests adjust horizontally and vertically for optimal positioning. Tool-free attachment to trunk support.



## Laterals

Slide in trunk support tracks with independent vertical, width, and angle adjustment. Swing out of the way for easy transfers.

**Small:** 4" x 5"

**Large:** 5" x 6"



## Push handle

Allows caregivers to assist riders and control tricycle speed. Available with or without caregiver brake.



## Rear steer

Lets caregivers assist with steering from behind, while giving riders a sense of independence. Available with or without caregiver brake.



## Hand anchors

Offer comfortable support for users with limited hand control or grip strength.



## Front handbrake

Provides speed control for riders. Adjustable for left or right-hand use. Includes parking lock feature.



## Front guide bar

Allows caregiver guidance and enables eye contact between the caregiver and rider.



## Abductor

Prevents leg scissoring by separating knees and controlling spasticity. Forward/back adjustment with depth markings.



#### **Abductor with adduction straps**

Adduction straps attach to the abductor pad, creating comfortable knee boundaries that maintain proper body and joint alignment.



#### **Pelvic harness**

Secures hips and upper thighs without abdominal pressure, providing stable positioning for postural control and proper spinal alignment. Controls extensor spasticity that causes sacral sitting.



#### **Pedal raise kit**

Accommodates leg length differences with stackable blocks. Use one or two blocks per side as needed.

**Block thickness:** ½"



#### **Stationary stand**

Converts tricycle into an indoor trainer for pedaling practice and all-weather exercise.



#### **Mounting bar**

Clips to the handlebar center and enables device mount attachment. Ideal for communication devices.



#### **Front basket**

A convenient storage space for small items.



#### **Outdoor cover**

Weather-resistant cover protects your tricycle during outdoor storage.



#### **Freewheel**

Allows caregivers to disconnect the pedals from the drivetrain so the tricycle can be pushed by the caregiver while the pedals remain stationary.

### **Need support?**

The Rifton website offers video tutorials, training resources, sample letters of medical necessity, evidence-based articles, and more.



Scan here to order, request a quote, or see detailed product information.