AVANOS

MIC-KEY® SINGLE PORT FEED EXTENSION SET WITH ENFit® CONNECTOR

Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.
Single Patient Use Only.

⚠ **Warning:** For enteral nutrition and/or medication only.

 Nonsterile	Not made with natural rubber latex	Product is NOT made with DEHP as a plasticizer	Rx Only
 Caution	Consult instructions for use	 2797	 0144-12 0144-24

Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA. In USA, 1-844-4AVANOS (1-844-428-2667). www.avanos.com
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
  Avanos Medical Belgium BVBA, Leonardo Da Vinciiaan 1, 1930 Zaventem, Belgium.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067 Australia.
*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2023-09-11 15-M1-331-02 / 50001872
www.mic-key.com

EN MIC-KEY® Single Port Feed Extension Set with SECUR-LOK® Right Angle Connector and Bolus ENFit® Connector and Clamp

For MIC-KEY® Low-Profile Gastrostomy Tubes Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

- 1.Remove the plug from the MIC-KEY® feeding tube to reveal the MIC-KEY® feeding tube anti-reflux valve.
 - 2.For gastric decompression or enteral feeding, connect the Extension Set connector by aligning the black line on the connector with the black line on the MIC-KEY®. Lock into place by rotating it CLOCKWISE 3/4 turn.
 - 3.Flush the Extension Set and the MIC-KEY® with warm water (10 ml for adults, 3–5 ml for children). Check for ease of flushing.
 - 4.After feeding or decompression, flush the Extension Set and the MIC-KEY® with 5–10 ml of warm water.
- 5.To disconnect the Extension Set from the MIC-KEY® rotate the extension set connector COUNTER-CLOCKWISE until the black line on the MIC-KEY® aligns with black line on the Extension Set. Gently remove the connector from the MIC-KEY®. Replace the plug into the MIC-KEY®.
- 6.After each use, wash the Extension Set with warm, soapy water and rinse thoroughly.

Device Lifetime

It is intended to be used multiple times for a single patient for up to 14 days.

Single Patient Use Only.

Not made with natural rubber latex.

Product is not made with DEHP as a plasticizer.

⚠ **Warning:** For enteral nutrition and/or medication only.

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